






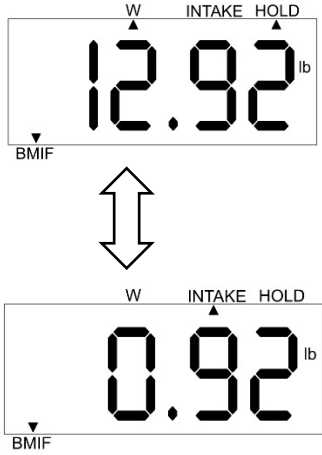


ADDENDUM

MB130

Breast Milk Intake Function

Turn Scale On	
Weigh baby before feeding	
Press HOLD key until displays STORE	
BMIF Annunciator will illuminate	
Remove baby from scale.	
After feeding place baby on scale	

<p>Press HOLD key until displays CHG</p>	
<p>Baby may be removed from scale Display will alternate between current baby weight and milk intake (difference from first weight)</p>	
<p>Press HOLD key until display exits BMIF function and returns to Zero in normal weighing mode.</p>	