

Instruction Sheet


Extended Life Battery #350716

Your Extended Life Battery should be fully charged before being put into operation. Included with the battery are both 12 Volt DC and 120 Volt AC chargers. Once plugged in, the first LED will show red during charging and will blink when fully charged. During operation the charge level can be checked by pressing the silver button. The number of LED's lit indicate the charge level.



After charging your battery, check the back of your device's charger to determine the proper output voltage. If the output is 18-20 Volts the switch should be left in the default 19 Volt position. If the output is 11-13 Volts, use a sharp pencil to move the switch to the 12 Volt position.

The battery can be connected to your device either directly by its cable or through the cable with one of the adapters jacks. The jack should fit snugly into your device's socket.

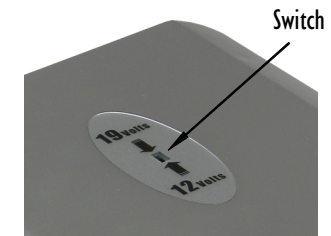

Other Notes

The battery cannot be charged while it is powering your device. Leaving the charger connected will cause no damage but will not maintain the battery power. To maintain the battery's full strength, a dormant battery should be fully discharged and then fully charged at least once a month.

MANUFACTURERS OF **IBeam**® PRODUCTS
OMNIMED INC.®