


Warranty Guidelines

Please read before using the rev•o•lu•tion™ case carts. Periodically clean cart as follows to preserve its luster and prolong its life.

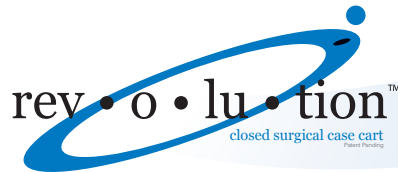

Stainless steel

- Stainless steel cleaner may be used on stainless portions of the cart only. Rinse with clean water and dry with chamois to prevent corrosion and water spotting.

▲ CAUTION ▲

To avoid scratching DO NOT USE stainless steel cleaner on powder coated white doors or polycarbonate windows.

Cart Body

(Including windows & bumpers)

- Wipe down with a mild soap (Joy dish soap recommended) and lukewarm water solution using a soft lint free cloth. Rinse with clean water and dry with chamois to prevent corrosion and water spotting.

White Bumper

- Scuffs and marks can be removed using Isopropyl Alcohol (rubbing) and a soft lint-free cloth.



Wheels

Lubrication is not possible on these maintenance-free **re•volve™** casters. Check wheels for visible points of tread wear. Check for any loose caster mounting bolts. Tighten to 100 to 105 in-lb.

Windows

- Made of scratch resistant, high-impact polycarbonate.
- Wipe down with a mild soap (Joy dish soap recommended) and lukewarm water solution using a soft lint free cloth. Rinse with clean water & dry with chamois to prevent water spotting. Formula 409 or Isopropyl (rubbing) Alcohol may also be used.

▲ CAUTION ▲

To avoid scratching windows DO NOT USE:

- Glass cleaner (Windex, etc.)
- Any cleaner with abrasives
- Abrasive cloths, squeegees or other cleaning implements on polycarbonate windows.

Replacement Parts

Casters, handles, doors, door latches & catches, windows, bumpers, can be removed and replaced if needed.

IMPORTANT NOTE

Lack of a regular maintenance schedule may void your warranty!