



Instructions for Use


Rectangle Major Procedure Table, Phenolic 800-0056

Rectangle Major Procedure Table, Carbon Fiber 800-0024

Replacement Deluxe Foam Pad (Rectangle Table) 508-0073

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.

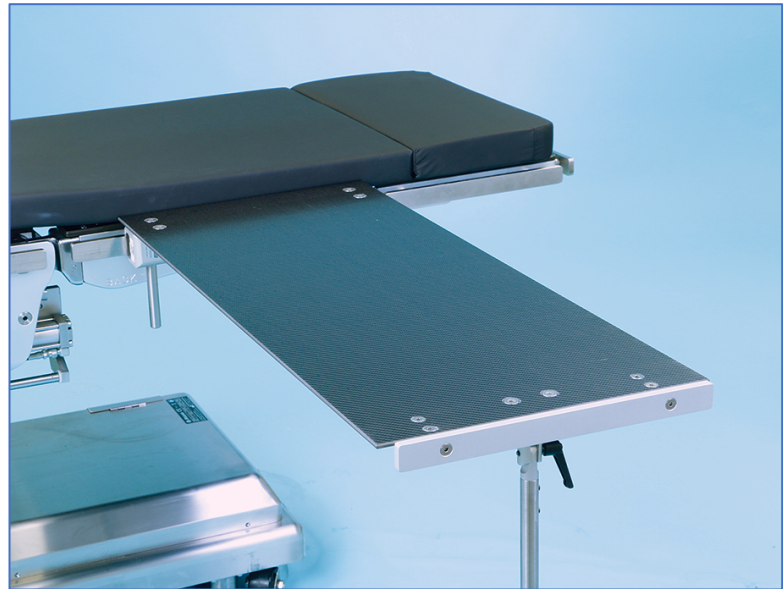

Initial Release: 9/8/2012



Manufacturer
SchureMed (081001460)



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

INTENDED USE

Its function is to provide a stable platform for all arm and hand surgeries.

GENERAL SPECIFICATIONS


Device Dimensions (maximum)

- *Length: 34" +/- 0.5"*
- *Width: 15" +/- 0.5"*
- *Depth: 42" +/- 0.5"*
- **Phenolic Device Weight: 15 +/- 0.5 lbs.**
- **Carbon Fiber Device Weight: 10 +/- 0.5 lbs.**
- *Attaches to surgical table anywhere on the rail using two rail clamps*
- *Single-person installation*
- *Clean with hospital grade disinfectant*

INSTRUCTIONS FOR USE

Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching Procedure Table to Surgical Table

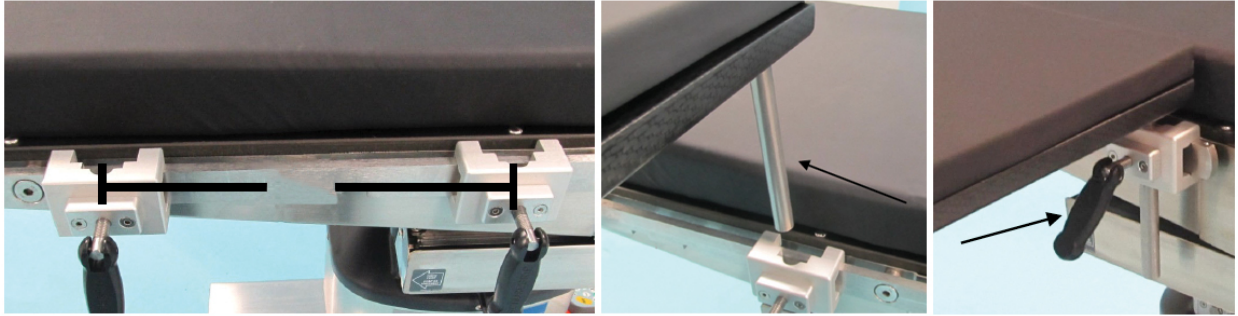

1. Attach Deluxe Rail Clamps (sold separately) to side rails and have them approximately 1' apart
2. Insert mounting rods into clamps
3. Adjust procedure table so that it is level with surgical table pad. Secure table to accessory rail using clamp locking knobs
4. Includes 1" pressure management pad
5. Clamps sold separately

Adjusting Table Leg



(1) End Rail • (2) Adjustable Swivel Leg Locking Handle • (3) Adjustable Height Locking Handle

1. Swivel leg in 360° to obtain unlimited C-arm axis (see above)
2. Loosen Adjustable Leg Handle (3) and extend leg until it sits firmly against floor then lock handle tight by turning it to the right
3. End rail accommodates traction attachments

COMPONENT OVERVIEW

Rectangle Major Procedure Table, Phenolic or Carbon Fiber, are surgical table extensions that attach to all rails.

Replacement Pad

508-0073 Deluxe Foam Rectangle Table Pad

Two Deluxe Rail Clamps Required for Use

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0248

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0248-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0248-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0248-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0248-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0248-SWISS

GENERAL INFORMATION

- Product not made with Natural Rubber Latex
- Device supports 500 lb. proportional patient load (6'4" tall patient per 99% human body model)
- Product warranty covers product from manufacturing defects for period of 2 years
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- Product is maintenance-free, check product condition before next use
- Life of device is 5 years under normal use
- Store device between -4°F to +86°F

DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



Product Use Warnings

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazards result from incorrect use. Strictly follow Instructions for Use with your Operating Table system.

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.



WARNING!

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*



Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community

IFU-800-0056 REV 3.01

IFU-800-0024 REV 3.01