



## Instructions for Use

**Hourglass Major Procedure Table, Phenolic 800-0040**

**Hourglass Major Procedure Table, Carbon Fiber 800-0031**

**Replacement Deluxe Foam Pad (Hourglass) 508-0106**

### Copyright Notice

*All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.*



**Initial Release:** 9/28/2006



**Manufacturer**  
**SchureMed** (081001460)



**Authorized Representative**

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

### INTENDED USE

Its function is to hold a patient's arm during surgery.

### GENERAL SPECIFICATIONS


*Device Dimensions (maximum)*

- *Length: 32" +/- 0.5"*
- *Width: 15" +/- 0.5" Middle*
- *Width: 23" +/- 0.5" Ends*
- *Depth: 23" +/- 0.5"*
- *Phenolic Device Weight: 18 +/- 0.5 lbs.*
- *Carbon Fiber Device Weight: 11 +/- 0.5 lbs.*
- *Attaches to rail of surgical table at any point on the rail using two rail clamps*
- *Single-person installation*
- *Clean with hospital grade disinfectant*

## INSTRUCTIONS FOR USE


Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

### Attaching Procedure Table to Surgical Table

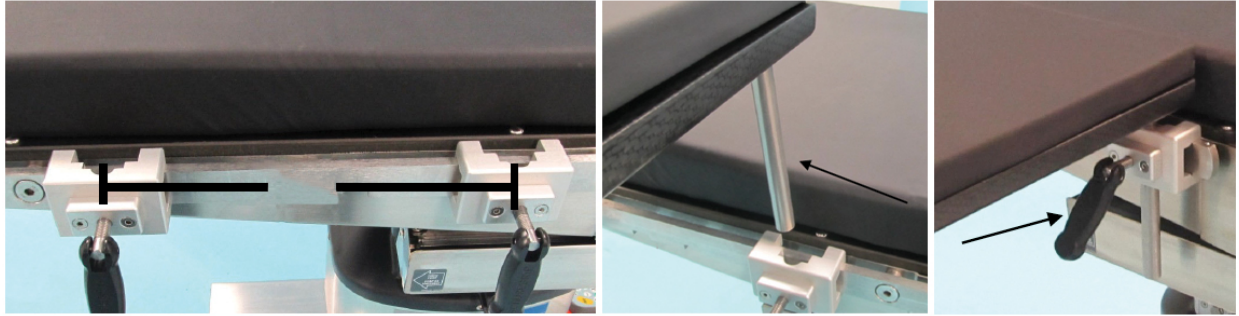

1. Attach Deluxe Rail Clamps (sold separately) to side rails and have them approximately 1' apart
2. Insert mounting rods into clamps
3. Adjust procedure table so that it is level with surgical table pad. Secure table to accessory rail using clamp locking knobs
4. Includes 1" pressure management pad

### Adjusting Table Leg

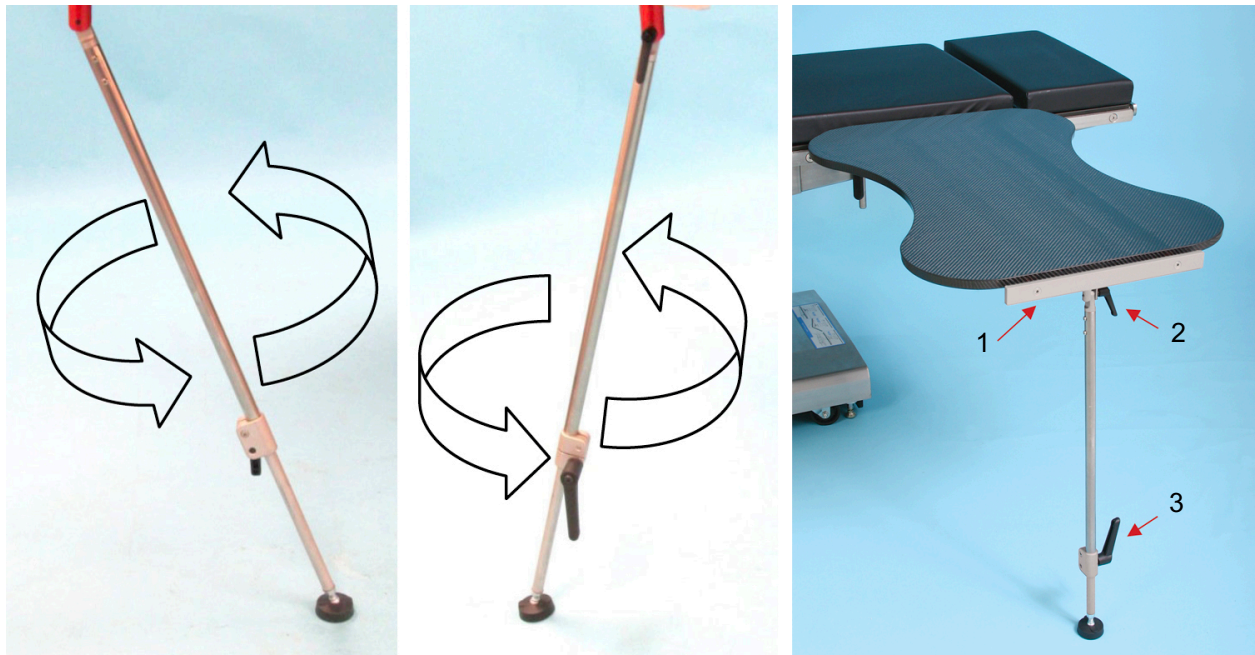

**(1) End Rail • (2) Adjustable Swivel Leg Locking Handle • (3) Adjustable Height Locking Handle**

1. Swivel leg in 360° to obtain unlimited C-arm axis (See above)
2. Loosen Adjustable Leg Handle (3) and extend leg until it sits firmly against floor then lock handle tight by turning it to the right
3. End rail accommodates traction attachments

## COMPONENT OVERVIEW

Hourglass Major Procedure Table, Phenolic or Carbon Fiber, are surgical table extensions that attach to all rails.

### Replacement Pad

508-0106 Deluxe Foam Hourglass Table Pad

### Two Deluxe Rail Clamps Required for Use

*US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0248*

*Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0248-DEN*

*Europe: 0.394" x 0.984" (10 mm x 25 mm ) PN# 800-0248-EU*

*Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0248-UK*

*Japan: 0.354" x 1.260" (9 mm x 32 mm ) PN# 800-0248-JPN*

*Swiss: 0.394" x 1.181" (10 mm x 30 mm ) PN# 800-0248-SWISS*

## GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. proportional patient load (6'4" tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F*

## DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



### Product Use Warnings

#### WARNING!

*Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*



#### WARNING!

*Hazard resulting from incorrect use. Strictly follow Instructions for Use with your Operating Table system.*



#### WARNING!

*Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory*

## CLEANING RECOMMENDATION

*Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.*

### CAUTION

*Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.*

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*



---

### Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative  
in European Community

IFU-800-0040 REV 3.01  
IFU-800-0031 REV 3.01