



## Instructions for Use

**Minor Procedure Table, Phenolic  
800-0026**

**Minor Procedure Table, Carbon Fiber  
800-0027**

**Replacement Pad  
Deluxe Foam Pad (Minor) 508-1432**

### Copyright Notice


*All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided for in relevant copyright laws*



**Initial Release:** 1/25/2012



**Manufacturer**  
SchureMed (081001460)

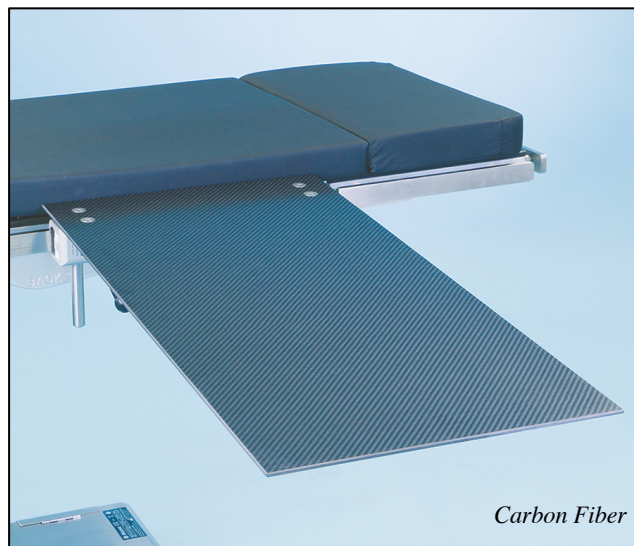

**EC REP** **Authorized Representative**

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

### GENERAL SPECIFICATIONS

*Device Dimensions (maximum)*


- *Patient Weight Capacity: 600 lbs. (20 lbs. @ far end of table, 41 lbs. in center of table)*
- *Length: 30" +/- 0.5"*
- *Width: 15" +/- 0.5"*
- *Maximum Raised Height: 3.5" +/- 0.5"*
- *Phenolic Depth: .375" +/- 0.5"*
- *Carbon Fiber Depth: .25" +/- 0.5"*
- *Phenolic: Device Weight: 10 +/- 0.5 lbs.*
- *Carbon Fiber: Device Weight: 9 +/- 0.5 lbs.*
- *Hand Table Mounting Post: 6"L*
- *Attaches to rail of surgical table at any point on rail*
- *Single-person installation*

## INTENDED USE

Its function is to provide a stable platform for all arm and hand surgeries.

## INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

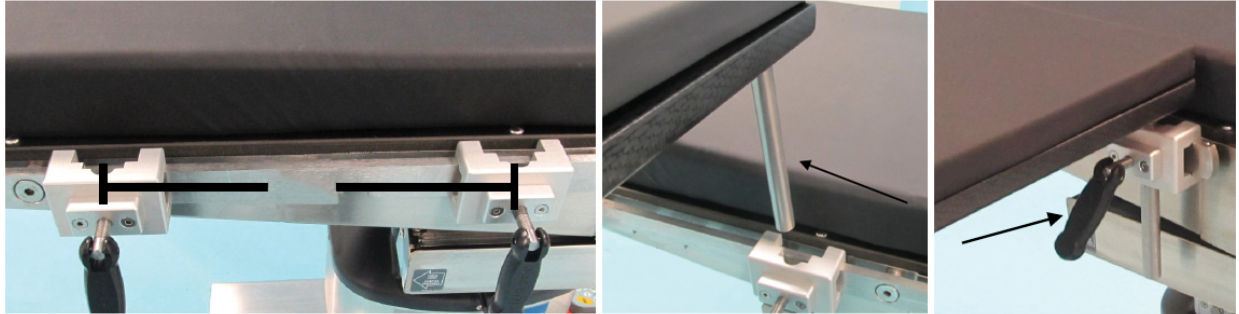

1. Attach Deluxe Rail Clamps to side rail and place them approximately 1' apart (as shown)
2. Insert mounting rods into clamps (as shown)
3. Adjust procedure table so that it is level with surgical table pad. Secure table to accessory rail using the Locking Knobs of the clamps (as shown)
4. Includes 1" pressure management pad
5. Clamps sold separately

## COMPONENT OVERVIEW

Rectangle Minor Procedure Table, Phenolic or Carbon Fiber are surgical table extensions that attach to all rails.

### Replacement Pad: 508-1432 Deluxe Foam Pad (Minor)

#### Deluxe Rail Clamps for use (2 required)

- **US:** 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0248
- **Denyer:** 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0248-DEN
- **Europe:** 0.394" x 0.984" (10 mm x 25 mm ) PN# 800-0248-EU
- **Eschmann (UK):** 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0248-UK
- **Japan:** 0.354" x 1.260" (9 mm x 32 mm ) PN# 800-0248-JPN
- **Swiss:** 0.394" x 1.181" (10 mm x 30 mm ) PN# 800-0248-SWISS

## GENERAL INFORMATION

- Product not made with Natural Rubber Latex
- Device supports 600 lb. proportional patient load (6'4" tall patient per 99% human body model)
- Product warranty covers product from manufacturing defects for period of 2 years
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- Product is maintenance-free, check product condition before next use
- Life of device is 3 years under normal use
- Store device between  $-4^{\circ}\text{F}$  to  $+86^{\circ}\text{F}$

## DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



### Product Use Warnings

#### WARNING!

*WARNING! Maximum load should not exceed appropriate proportion of a patient weighing 600 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*



#### WARNING!

*Hazard resulting from incorrect use. Strictly follow Instructions for Use with your Operating Table system.*

## CLEANING RECOMMENDATION

*Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.*



#### WARNING!

*Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.*

## CAUTION

**Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.**

1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.
2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory
3. Let accessory dry



---

## Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer

**LOT**

Batch Code/Serial Number

**EC REP**

Authorized Representative  
in European Community

IFU-800-0026 REV 3.02  
IFU-800-0027 REV 3.02