



Instructions for Use

Schure Spine Frame Dolly 800-0074-SF

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.



Initial Release: 6/12/2012



Manufacturer
SchureMed (081001460)



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands



GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to $+86^{\circ}\text{F}$*

GENERAL SPECIFICATIONS


Device Dimensions (maximum)

- *Length: 27" +/- 0.5"*
- *Width: 12" +/- 0.5"*
- *Height: 39" +/- 0.5"*
- *Device Weight: 28 +/- 0.5 lbs.*
- *Fully Assembled*

INTENDED USE

Intended use is to store components of our Schure Spine Frame.

INSTRUCTIONS

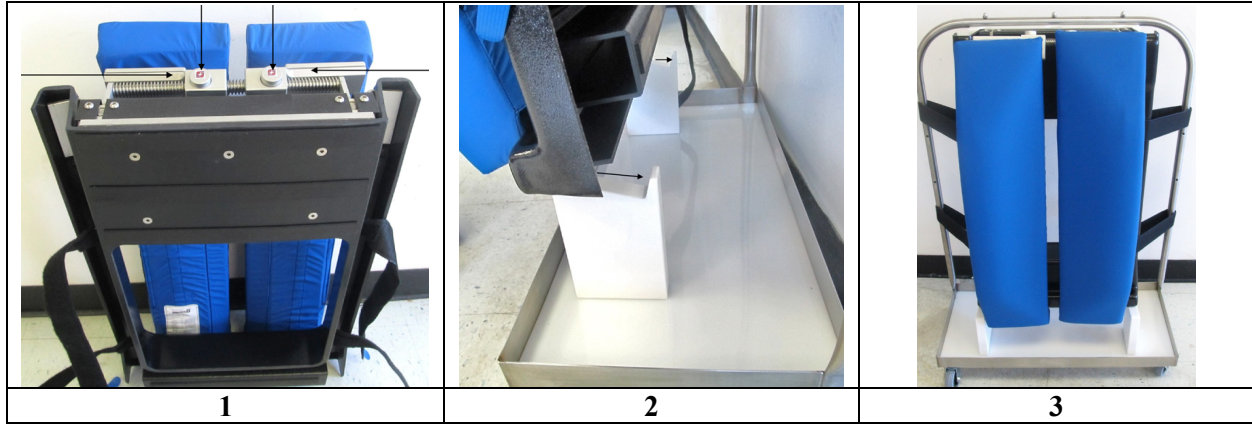

Step 1. When placing Spine Frame onto Dolly, have buttons facing up and close together as shown.

Step 2. Place Spine Frame onto Dolly (as shown) and slide all the way back until plastic of frame aligns with back of Dolly blocks.

Step 3. Loop velcro straps around Dolly and fasten them onto frame.

COMPONENT OVERVIEW

Schure Spine Frame Dolly is a storage cart used to store components of our Schure Spine Frame.

DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- **SchureMed accepts back used or retired products** - or dispose of product in accordance with national requirements



PRODUCT USE WARNINGS

WARNING!

Hazard resulting from incorrect use. Be absolutely sure to follow Instructions for Use for when operating Dolly.

CLEANING & DISINFECTING

Spray and wipe clean with hospital approved disinfectant. No sterilization.



Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community

IFU-800-0074-SF REV 3.02