



Instructions for Use

Disposable Face Masks 800-0164

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.



Initial Release: 4/16/2010



Manufacturer
SchureMed (081001460)



EC REP Authorized Representative

Emergo Europe, Prinsessegracht 20
2514 AP The Hague, The Netherlands

INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

1. Place patient on surgical table/shoulder chair in supine position
2. After intubating patient, hold patient's head/neck and raise shoulder chair up so patient is in Fowler (sitting) position
3. Loosen ball joint and adjust head positioner around patient's head. Make sure patient's ears are uncovered. Then tighten ball joint.
4. Holding head positioner flaps alongside patient's head, place face mask over face as shown - making sure eyes, nose and mouth are uncovered
5. Affix top and bottom velcro straps of mask on one side to head positioner. Then affix other side as shown - making sure facemask is centered over face.

NOTE: Head should be securely positioned. If not, make straps a little tighter.

INTENDED USE

Intended use is to hold patient's head during operative procedure.

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- Length: 29" +/- 0.5" (Strap), 9" +/-0.5" (Face Mask)
- Width: 13.5" +/-0.5"
- Depth: 1" +/-0.5"
- Device Weight: 0.25 +/- 0.5 lbs.
- Attaches to head section of SchureMed Shoulder Chairs
- Single-person installation

COMPONENT OVERVIEW

Disposable Face Mask securely holds a patient's head during operative procedure. Straps are soft and pliable to attach to head frame.

GENERAL INFORMATION

- Product not made with Natural Rubber Latex
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- Store device between $-4^{\circ}F$ to $+86^{\circ}F$

SINGLE PATIENT USE—Please dispose after each use



Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community

IFU-800-0164 REV 3.03