

# INSTRUCTION MANUAL

## Pediatric Water Rescue Manikin

149-1350 – Rescue Cathy (Newborn)

149-1352 – Rescue Billy (6- to 9-Month-Old)

149-1351 – Rescue Timmy (3 -Year-Old)



# About the Simulator

Thank you for purchasing a Pediatric Water Rescue Manikin. The product comes with two tools; a faucet connection hose and an air inflation bulb. To simulate realistic weight and to use in water rescue and safety training, fill manikin with water using the following instructions.



## FILL MANIKIN

Steps:

1. Hold manikin upside down or lay in sink.
2. Open both valves on bottom of manikin's feet.

\*Rescue Timmy has one metal valve on the left foot and one aqua drain valve with an internal flap on the right foot.

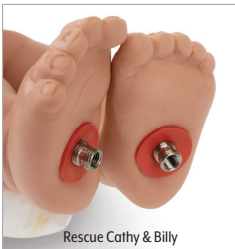


Figure 1

3. Attach hose from faucet to left metal valve on baby's foot or insert through internal flap of aqua drain valve on Timmy's foot. (See Figure 1.)
4. Fill manikin with warm water until water begins to flow from the open metal valve on right foot.
5. Shut off flow of water.
6. Close valve on right foot.
7. Remove hose from valve on left foot.
8. Gently squeeze air from manikin until water flows from left valve. Close valve tightly.
9. Let manikin lay 1 or 2 minutes with head down.
10. Repeat steps 2 through 8 making sure all air is out of the manikin.

**NOTE:** Do not store manikins filled with water. Microbiological growth could occur and extreme temperature will effect the manikin

# Instructions for Use

## DRAIN MANIKIN

After every use the manikin must be drained of all water and filled with air before storing it.

Steps:

1. Place manikin's feet into sink and open both valves. Insert pencil or dowel (DO NOT USE PEN) through internal flap of aqua drain valve for Timmy manikin only. (See Figure 2.)
2. Let drain 30 minutes.
3. Holding manikin over sink, squeeze excess water from manikin.
4. Repeat steps 2 and 3.
5. Close metal valve on left foot of baby manikin. Close aqua drain valve cap on Timmy manikin.
6. Attach tubing from inflation bulb to metal on right foot. Inflate manikin with air by pumping bulb.

### **DO NOT OVERINFLATE.**

Grasp valve with tubing and turn to close. (See Figure 3.)

7. Remove inflation bulb assembly.



Figure 2



Figure 3

## PART NUMBERS

149-1350 Rescue Cathy  
Newborn Size

149-1352 Rescue Billy  
6- to 9-Month-Old

100-2252 Carry Bag for  
Rescue Cathy and Billy

149-1351 Rescue Timmy  
3-Year-Old

100-2260 Carry Bag for Rescue Timmy

