



## THANK YOU FOR CHOOSING SCAN-GUARD®.

PLEASE READ INSTRUCTIONS BEFORE USE.

### INDICATIONS FOR USE

Scan-Guard® is an equipment protection cover intended to act as a barrier against fluids.

It is made from durable materials that inhibit bacterial and fungal growth. When dimensions are measured accurately, Scan-Guard® should easily slide over the table pad and attach to the cradle using the existing hook or loop adhesive. If hook or loop adhesive is needed, please let us know and we will provide it for you. This product is latex-free, hypoallergenic and promotes stain and fluid resistance.

### CLEANING

Abrasive cleaning agents are not to be used. Clients should pre-test a small area of fabric to guarantee that Scan-Guard Protective covers perform as expected. The cover is not sterile when shipped. We suggest you sanitize it thoroughly upon arrival, and after each use, with a bleach/water solution as recommended by the Center for Disease Control or other controlling authority.

### ROUTINE CARE

To disinfect, wipe the surface with a neutral detergent and lukewarm water. You may also use a hospital grade disinfectant. Rinse thoroughly with water and dry completely before next use. Scan-Guard® can be rolled or folded when not on the imaging machine.

### TROUBLESHOOTING

Do not use if torn or damaged. For comments or concerns regarding your purchase, please contact Customer Service.

### LIMITED WARRANTY

Products are warranted to the original purchaser to be free from manufacturer defects in materials and workmanship for a period of 90 days from the original purchase date. If your product does not work properly because of a defect in materials or workmanship, the Company will, for the length of the warranty period, at its option, either (a) repair your product with new or refurbished parts, (b) replace it with a new or a refurbished equivalent value product, or (c) refund your purchase price. All returns and repairs require proof of purchase and a return authorization number. A warranty return and/or repair must be shipped freight prepaid and insured to the address provided by the Company at the time of approval of the warranty claim. The decision to repair, replace, or refund will be made by the Company. Warranty claims may only be processed through the Company or an authorized supplier approved by the Company. This warranty is non-assignable.

### THE COMPANY HEREBY DISCLAIMS ANY AND ALL OTHER WARRANTIES, REPRESENTATIONS, AND GUARANTEES, EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR USE OR PURPOSE.

The warranty set forth herein is the sole and exclusive warranty and remedy provided by the Company. Damage caused by improper set-up or use other than in accordance with Company provided instructions, accident, abuse, neglect, normal wear and tear, or exposure to extreme temperature, humidity, or weather are excluded. Any modifications to the product, or unauthorized repairs, invalidate the warranty and immediately terminate all liability of the Company. Failure to follow recommended preventative maintenance or instructions for product care may void the warranty. In no event shall the Company be liable for any special, direct, indirect, incidental, exemplary, punitive, loss of profit, or consequential damages or costs.



FOR SAFER, CLEANER, MORE COMFORTABLE MEDICAL IMAGING



## INSTALLATION INSTRUCTIONS

1. Open the plastic sleeve containing Scan-Guard®.
2. Remove the table pad from the imaging machine and thoroughly clean the cradle and table pad.
3. Once the cradle and table pad are dry, place the table pad inside the Scan-Guard® using the open hole in the bottom to ease the table pad into the cover.
4. Close the end flaps.
5. Line up the encased table pad with the hook or loop adhesive on the cradle and apply pressure to ensure the table pad and cover are securely attached to the cradle.
6. Scan-Guard® flaps should be facing downward to ensure liquids fall to the floor and do not pool inside.
7. If you have also purchased the extender cover, repeat the process above.




**STEP 1** Turn table pad upside down and begin sliding table pad inside the cover.



**STEP 2** Use the opening on the underneath side to help the pad move through the cover.



**STEP 3** Close the end of the cover with hook and loop closure.

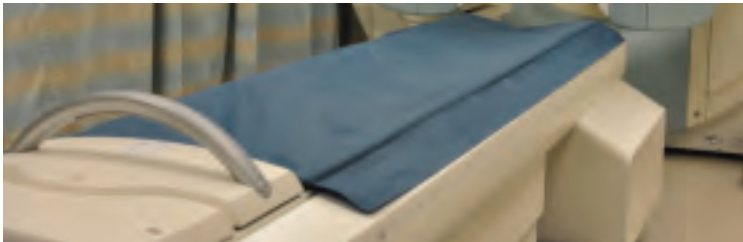

**STEP 4** Turn pad over and allow loop/hook of cover to attach to Hook/loop on the table cradle.

**MR** ALL PRODUCTS MEET  
MR SAFE CLASSIFICATION



**WARNING:** Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

FOR SAFER, CLEANER, MORE COMFORTABLE MEDICAL IMAGING