



Be  
**READY**

DANi Teen & Young Adult Advanced Training Solution

*Nasco*  
HEALTHCARE



# Quick Start Guide

DANi™ is packaged with arms and female genitalia attached. Legs have been disconnected to prevent possible damage during shipping.

1. Carefully unpack the product's individual components (see Fig. 1). Your DANi manikin includes:

- Head, torso, arms and female genitalia
- Left leg
- Right leg
- Male genitalia
- Accessories

2. Connect left and right legs to the torso (see Fig. 2).

3. Female genitalia attached, or connect male genitalia (see Fig 3 & 4).

4. Your product is ready for use.

DANi comes pre-assembled to support nasogastric tube procedures (lavage, gavage). Please see section Oral and Nasal Lavage of the instruction manual if you want to configure your product to support different airway management procedures.



Figure 1

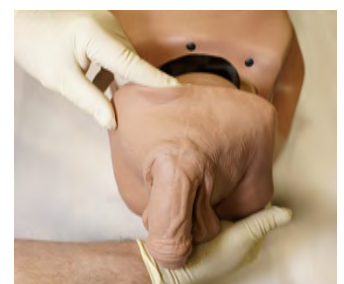


Figure 3



Figure 2

Figure 4