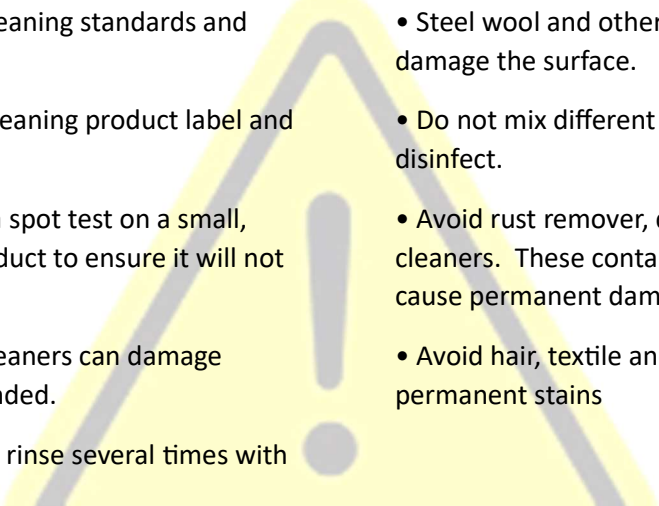


CLEANING INSTRUCTIONS

WARNING!

- 
- Always comply with facility cleaning standards and guidelines* .
 - Always follow directions on cleaning product label and heed warnings.
 - Before using any cleaner, do a spot test on a small, inconspicuous area of your product to ensure it will not cause damage to the finish.
 - Solvents, acidic or abrasive cleaners can damage surfaces and are not recommended.
 - Wipe spill away promptly and rinse several times with water.
 - Steel wool and other abrasive cleaning pads will damage the surface.
 - Do not mix different cleaning agents to clean or disinfect.
 - Avoid rust remover, drain cleaners and toilet bowl cleaners. These contain harsh chemicals and will quickly cause permanent damage.
 - Avoid hair, textile and food dyes, as they can cause permanent stains

*For additional information please refer to the CDC cleaning and disinfection protocols.

RECOMMENDED CLEANING PRODUCTS

Soap and water and most common cleaners/disinfectants are excellent for most cleaning and disinfecting purposes as long as the directions on the label are followed. The following cleaning products are acceptable when used according to directions:

- | | | |
|---|-----------------------|---------------------|
| • AHP Accel® TB | • Hydrogen Peroxide | • Sani-Cloth® AF3 |
| • Bleach Mixture (1/3 cup bleach per 1 gallon of water) | • Isopropyl Alcohol | • Sani-Cloth® HB |
| • Dispatch® Hospital Cleaner Disinfectant | • Metrex™ CaviWipes™ | • Sanimaster™ 6 |
| • Diversey™ Oxivir® Tb Wipes | • Oxivir® Tb | • Super Sani-Cloth® |
| | • Quaternary Cleaners | |

RECOMMENDED CLEANING PROCESS

1. Wipe up any spills and remove any debris before applying cleaner/disinfectant.
2. Apply cleaner/disinfectant according to directions on cleaning agent label.
3. Avoid using sprays on products with electronic components.
4. Ensure that cleaning solution is wiped away promptly, directions on label.
5. Dispose used cleaning cloths and solution appropriately.