



# HOW TO INSTALL

## RADIATION SHIELDING PLEATED TABLE SHIELD

COPYRIGHT ©2025. ALL RIGHTS RESERVED

**HOW TO INSTALL**  
RADIATION SHIELDING PLEATED TABLE SHIELD

**STEPS**

**ATTENTION**

BEFORE INSTALL THE RADIATION SHIELDING PLEATED TABLE SHIELD, YOUR IMAGING TABLE MUST BE CLEAN AND DRY.

WE RECOMMEND CLEANING YOUR IMAGING TABLE WITH CAVI WIPES.

1

WITH THE NON-SLIP PAD FACING THE IMAGING TABLE. PEEL THE PROTECTIVE FILM



**HOW TO INSTALL**  
RADIATION SHIELDING PLEATED TABLE SHIELD

**STEPS**

**2**

**POSITION THE LEAD CURTAIN  
ALIGNED WITH THE CENTER OF  
YOUR TABLE.**



**REMEMBER**  
The table surface  
**MUST BE** completely  
clean and dry.

**3**

**ATTACH THE ENTIRE SURFACE  
WITH THE TABLE.**



4

**PRESS FIRMLY TO ENSURE  
ADHERANCE.**

