

Plastic Waste Bin

Dimensional Illustrations and Installation Instructions



SKU Number: 1786593 – Plastic Waste Bin for M38 and M40 Carts
1874847 – Plastic Waste Bin for CareLink Carts

Product Description

The Plastic Waste Bin provides a convenient and sanitary way to quickly dispose of waste items.

Features and Benefits

- Dedicated placement of the Plastic Waste Bin makes waste elimination simple for cart users.
- Rolled rims provide added strength.
- Size and location provide a space-efficient solution to small, immediate waste removal.
- Plastic construction offers an easy-to-clean solution.

Specifications

1786593: Accepted on all M38, M38 XP, M38 RX and M40 Carts.

Dimensions: 10.33"h x 9.88"w x 6.75"d

1874847: Accepted on all CareLink™ Carts


Dimensions: 10.33"h x 9.88"w x 8.18"d

*Depth increase is due to spacer.


Bin Weight: .6 lbs

Bin Holder Weight: 1.6 lbs


Total Weight: 2.2 lbs

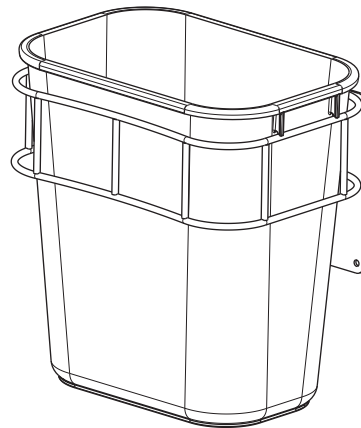

3D View

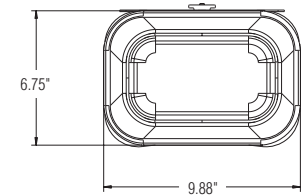

Top View

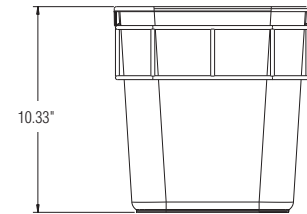

Front View

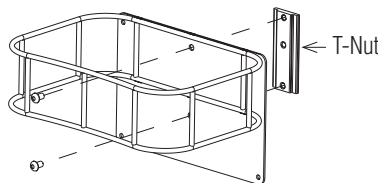


Figure 1
Step 4: Screw t-nut onto back of Holder.

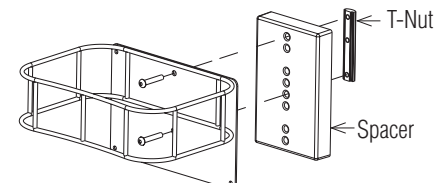


Figure 2
Step 4: Screw t-nut and spacer onto back of Holder.

Installation Instructions


1. Carefully remove the rear shuttle cap.
2. If desired mounting location is below the handle assembly, continue to Step 3. If desired location is above the handle assembly, skip to Step 4.
3. Remove the rear handle assembly.
 - a. Loosen the screws holding the rear handle in place using a 4mm hex driver.
 - b. Remove the handle assembly by sliding the assembly up the t-track; set aside.
4. Install the Plastic Waste Bin.
 - a. Screw t-nut onto the back of the Waste Bin Holder (see Figure 1). If securing SKU 1874847 on the CareLink column, screw t-nut onto spacer, and spacer onto the back of Waste Bin Holder (see Figure 2).
 - b. Slide the t-nut and bracket down until it rests at the desired location. If using SKU 1874847, must first install t-nut onto spacer (see Figure 2).
 - c. Secure the location on the t-track by tightening the screws.
5. Slide Plastic Waste Bin into the secured holder.
6. Replace the rear handle assembly.
7. Replace shuttle cap.

Note: If mounting the Waste Bin directly onto the side of the medication drawer, the t-nut is not needed. Use the four plastite screws provided to secure the Waste Bin Holder onto the side of the drawer.


Specifications subject to change without notice.

Mounting Location

- M38 XP, M38 RX, CareLink XP, and CareLink RX: Back t-track or directly onto side of the medication drawer
- M38 and CareLink: Option of front or back t-tracks

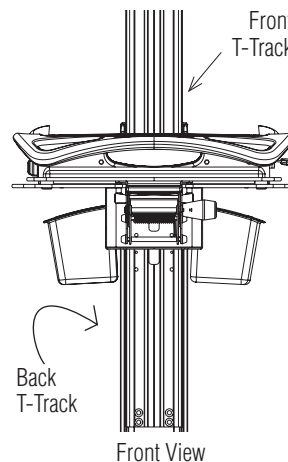

Front View

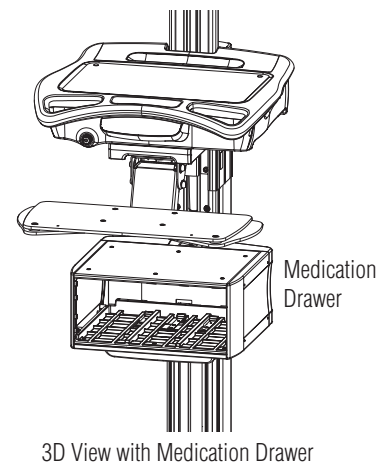

3D View with Medication Drawer