

# PRESSURECARE CLEANING SUMMARY REPORT

The Trulife range is easy to clean/ disinfect between surgical cases and is more hygienic than many disposable foam options which can facilitate bacterial growth by absorbing fluids. The disinfectant market is a very diverse one comprised of a number of suppliers. Thus, testing types of chemistries, rather than brands is the most logical approach. If there is something specific that you wish to confirm please contact your local distributor.

## DISINFECTING AND CLEANING RECCOMENDATIONS

Oasis + is autoclavable

Validated through testing with no discernible change in appearance, no degradation and no contaminations from the products after 24 Autoclave Sterilisation Cycles at 134 °C and 3 minutes.



Oasis, Oasis Elite, Azure, ARDS/ Elite foam positioners



Main Chemistry	Popular Cleaner (manufacturer)
Alcohol	Mikrozyd
Chlorine	Milton
Hydrogen Peroxide	Incidin OxyFoam S
Ammonia	Sani-Cloth AF Universal Wipe
Sodium Hydroxide	Terminator
Sodium Gluconate	Sani- Cloth detergent Wipes

Please note that it is not advisable to soak the Oasis, Oasis+, Elite and Azure pads in a cleaning solution overnight as this may affect the long-term durability of the products. Products should be wiped clean only. Always follow manufacturing guidelines for cleaning solution dilution ratios. Cleaning agents deemed corrosive to the skin should not be used on products that will come in contact with a patients skin.