

Healthometer<sup>®</sup>  
Professional



349KLX interfaces with

**WelchAllyn<sup>®</sup>**

Spot Vital Signs<sup>®</sup> LXi

Advantages

- Interfaces with Welch Allyn<sup>®</sup> Spot Vital Signs<sup>®</sup> Monitor**  
Scale has to ability to transfer weight data to Welch Ally Spot Vital Signs monitors via serial port connection
- Remote Display**  
The remote display provides versatility with the ability to be mounted on a wall or placed on a table
- Low Profile Platform**  
Low platform height requires only minimal step up, offering patient a more comfortable and stable weighing experience
- Value-Added Features**  
Operating functions (LB/KG Conversion, LB/KG Lock Out, Auto Zero, Zero Out/Tare, Auto Off, Parent/Child Weighing Function) included at no additional cost

**Health o meter<sup>®</sup>**

Professional

[www.homscales.com](http://www.homscales.com)

See reverse side for additional specifications



Model	Shipping Dims. (L x W x H) / Cube	Weight (LB)	UPC Code
349KLX	20.25" x 13.5" x 6.3" / 0.99	10.4 lb	892076002251

**Specifications**

- Capacity: 400 lb / 180 kg
- Resolution: 0.2 lb / 0.1 kg
- Platform Size: 12 ½" (w) x 12" (d) x 1 ⅞" (h) / 318 mm (w) x 305 mm (d) x 48 mm (h)
- Display: 1" LCD
- Connectivity: Serial Port
- Power Source: 6 AA batteries included, optional power adapter (ADPT40)
- Optional Carrying Case: Item #64771
- Functions: LB / KG Conversion, LB / KG Lock Out, Zero Out / Tare, Auto Zero, Auto Off, Parent / Child Weighing Function
- 2 Year Limited Warranty

Carrying Case for 349KLX  
Item# 64771



349KLX interfaces with  
Spot Vital Signs® LXi



Note:

When connecting the Welch Allyn® Spot Vital Signs® LXi to a Health o meter® Professional 349KLX Scale:

- Caution: Only use a 6 AA batteries with the Health o meter® Professional 349KLX scale. Do not use, the optional wall mounted power supply adapter.
- Caution: The Health o meter® Professional 349KLX scale, is not sold with an RS-232 cable to connect the scale to the Spot LXi. The customer must purchase a standard RS-232 cable.
- Caution: Remove the two hex nuts on the RS-232 cable, before screwing the cable into the scale base, when using the Spot Vital Signs® LXi with a Health o meter® Professional 349KLX scale.