Health o meter®

P r o f e s s i o n a l

600KL / 600KG Heavy Duty Eye Level Digital Scale With Integral Digital Height Rod

ISO13485:2016
CERTIFIED

ISO 9001:2015
CERTIFIED

E112697
UL 60601-1
CSA C22.2 No. 601.1-M90

Patent: D667323

USER INSTRUCTIONS

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

www.homscales.com

weigh**easier**®

Health o meter® Professional 600KL
Health o meter® Professional 600KG
© Pelstar, LLC 2020

Thank you for purchasing this Health o meter® Professional scale. Please read this manual carefully and keep it for easy reference or training.

TABLE OF CONTENTS

CAUTIONS AND WARNINGS.....	3
SPECIFICATIONS / DEFINITION OF SYMBOLS	4
CONNECTIVITY / CERTIFICATIONS / DISPOSAL	5
ASSEMBLY INSTRUCTIONS	6
SET-UP INSTRUCTIONS	10
OPERATING INSTRUCTIONS	11
MAINTENANCE AND CLEANING	14
CALIBRATION	15
TROUBLESHOOTING	16
HEIGHT ROD TROUBLESHOOTING	17
WARRANTY.....	18

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTIONS AND WARNINGS

INTENDED USE

This Health o meter® Professional platform scale is intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh patients who are safely positioned and standing on the platform. The intended use of the built-in height rod is to measure patient height. Do not modify the product or use it for anything other than its intended purpose.

To prevent patient/caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not wheel or pull the scale down stairs, doing so may damage internal parts.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.

Patient/caregiver safety

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer.
- To prevent patient injury, the patient must be attended throughout the entire weighing event. Caregivers should ensure that the patient is stable and provide support as needed when getting onto and off of the scale.
- Patient should wear socks or other form of lightweight foot covering during weighing event.

When operating the scale with the AC adapter:

- Operate this scale exclusively with the AC adapter that is provided (ADPT50). Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

When operating the scale with batteries:

- If the scale's battery icon indicates that the batteries are low replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

This equipment has been tested and found to comply with the EMC limits for the Medical Device Directive 93/42/EEC (EN 55011 Class A and EN 60601-1-2). These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates, uses and may radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. There is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to eliminate the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.
- Consult Customer Service or a field service technician for help.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries whatsoever arising out of or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS

600KL/600KG Scale Specifications

Health o meter® Professional model 600KL/600KG uses sophisticated microprocessor technology. This precision instrument is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient. The scale is set up to determine the weight of a stationary patient. The weight can be displayed in pounds or in kilograms. The 600KL/600KG features a swivel display which allows for viewing from either side of the scale plus wheels for easy mobility. The scale can be operated with a power adapter or by 6-C cell batteries (not included), depending on use.

Specifications

Capacity and resolution	600KL: 600 lb x 0.2 lb (272 kg x 0.1 kg) 600KG: 272 kg x 0.1 kg
Power Requirements:	Adapter Model UES06W0CP-060050SPA or UE15WCP-060050SPA (Health o meter Professional Model ADPT50) (INCLUDED), Input: 100-240V – 50/60Hz, 500mA Output: 6.0V  0.5A or 6 C-cell 1.5V batteries (not included)
Environmental:	Operating Temperature: 10°C to 40° C (50°F to 104°F) Storage Temperature: 0°c to 50°C (32°F to 122°F) Maximum Humidity: 85% RH in use or in storage
Platform Dimensions:	14 1/4" x 14 1/4" x 2 5/8" (36cm x 36cm x 6.8cm)
Assembled Dimensions:	14 3/4" x 24 1/8" x 54 3/8" (37.5cm x 61.3cm x 138cm)

Definition of Symbols

Attention: Consult Accompanying Documents

Caution

Do not dispose of this device in the domestic waste stream.

Type B Applied Parts

DC Power Supply

The scale surface, along with the height rod, (if provided) are the Type B applied parts.

CERTIFICATIONS / CONNECTIVITY / DISPOSAL

Certification Descriptions:

	Complies with ANSI/AAMI ES60601-1 (2005) + AMD 1 (2012, CAN/CSA-C22.2 No. 60601-1 (2014)
 	Pelstar, LLC has been officially certified as a manufacturer of Health o meter® Professional medical devices to be in compliance with stringent ISO requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

All Health o meter® Professional scales are manufactured in FDA-registered factories.

Scale Connectivity Information:

To enable reliable transmission of weight, height, and Body Mass Index (BMI) data, this scale is designed to connect to a computer, monitor, or other electronic device via USB or Pelstar® wireless technology. This connectivity helps minimize a source of potential medical errors caused by incorrectly copying and then recording patient data. For more information regarding how to connect the scale to other electronic devices, please contact Health o meter® Professional Scales Technical Support, available during normal business hours at 1-800-638-3722.

Note: If this product was purchased as a “BT” model, refer to the Wireless Communication Set Up Instructions included with the packaging.

Scale Disposal:

Do not dispose of this device in domestic waste. This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal. For scale assistance or telephone support, please call Health o meter® Professional Customer Service (available on normal business days) at (+1) 800-815-6615 or (+1) 708-377-0600.

ASSEMBLY INSTRUCTIONS

Before Assembly

Each 600KL/600KG Heavy Duty Eye Level Digital Scale with Integral Digital Height Rod scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

Carton

(1) Large brown box containing:

- (1) Scale platform

(1) Long brown box containing:

- (1) Pillar

(1) Small brown box containing:

- (1) Small white box containing the AC adapter (ADPT 50)
- (1) Plastic bag containing:
 - (1) 5mm Allen wrench
 - (4) 5mm hex screws (Allen screws)

(1) Medium brown box containing:

- (1) Display assembly
- (1) User Instructions

Tools Required

5mm Allen Wrench (included), Phillips Screwdriver (not included)


ASSEMBLY INSTRUCTIONS (CONT.)

Prior to assembly, remove all brown cardboard in and around pillar, base, and display head.

1. Carefully remove the contents of the carton and place each part on a flat, level, and dry surface. Place the scale platform on its side and align the pillar perpendicular to the platform. The front of the display head on the pillar should be facing forward and above the scale platform.


2. Locate the load cell cable connector at the bottom of the pillar. Feed the cable through the opening at the base of the platform.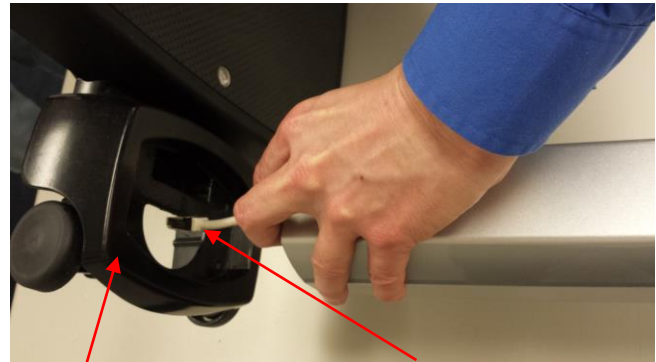

Platform Base

Load Cell Cable Connector


3. Insert the pillar into the pillar base on the platform. Insert the 4 hex screws on the underside of the platform and tighten using the supplied 5mm Allen wrench.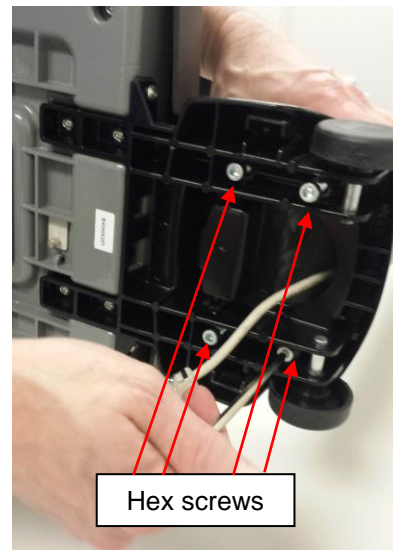

Hex screws


ASSEMBLY INSTRUCTIONS (CONT.)

4. Plug the load cell cable connector into the load cell plug in the platform base. The base of the scale has been designed to provide wire management for the load cell cable. Route the load cell cable as shown.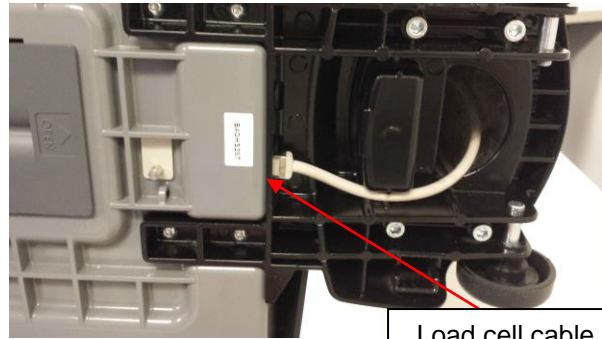

Load cell cable port


5. After plugging in the load cell cable, slide the section of plastic tubing surrounding the cable upwards.

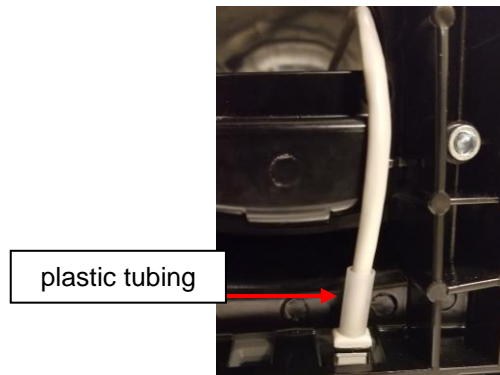plastic tubing

6. Tuck the cable and tubing into the protective guide in the underside of the scale platform. This holds the cable in place and protects it from being damaged.

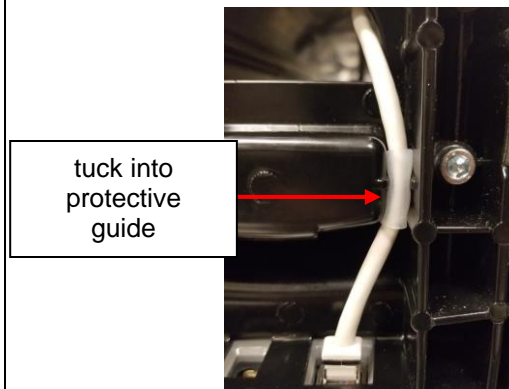tuck into
protective
guide

ASSEMBLY INSTRUCTIONS (CONT.)

7. Set the base/pillar assembly upright. Obtain the display head module. Plug the cable labeled "SCALE" into the SCALE jack in the display module. Plug the cable labeled "COM" into the COM jack in the display module.

8. Place the display module on top of the pillar and align the tab on the front of the pillar with the slot at the bottom of the display module.

9. Remove the screw from the back of the display module, place the module on top of the pillar and use a Phillips screw driver to replace and tighten the screw securing the display head to the pillar.

SET-UP INSTRUCTIONS

Set Up

1. Remove the protective film from the display window on the display head.
2. Ensure the scale base is on a stable, level and flat surface and that the sides of the base are clear of any obstacles.
3. Utilizing the leveling bubble located at the top of the scale platform, level the scale by adjusting the leveling feet at each corner of the underside of the base.

Powering the Scale

1. For AC operation, the scale offers two areas to attach the AC adapter. The AC adapter can be plugged into the base of the scale and into the power source. The AC adapter can also be plugged into the “DC IN” plug on the back of the display module and then into the power source.

Note: The Auto Off function is only active when the scale is being used without the AC adapter.

2. For battery operation, see Battery Installation below.

Battery Installation

1. Place the scale on its side. The battery holder is located in the center of the bottom of the base.
2. Slide the battery cover in the direction of the “open” arrow.
3. Place the 6 C-cell batteries into the battery holder. Note the orientation that is shown and ensure the batteries are placed in the proper direction. Health o meter® Professional recommends using Alkaline batteries. Do not use zinc-carbon batteries.
4. Replace the battery cover.

Note: When powering the scale via batteries, the AC adapter must be completely disconnected from the scale.

Battery compartment


(underside of scale platform)

Warning: If the scale will not be used for some time, remove all batteries to avoid a safety hazard. Disposal of batteries should be done according to the national, regional or local regulations which apply to you for disposal of batteries.

OPERATING INSTRUCTIONS

Using the Keypad

KEY	FUNCTION
ON/OFF	Power scale ON and OFF.
ZERO	Zeroes the scale prior to weighing.
HOLD/RELEASE	Holds the value of the weighed object on the display until the button is pressed again to clear the value.
UNITS	600KL Only: Toggles between kilograms or pounds.
▲	Scrolling method for entering weight when in Calibration mode.
WEIGHT/HEIGHT/BMI	Toggles the display between the weight reading, height reading and corresponding Body Mass Index (BMI) reading.
▼	Scrolling method for entering weight when in Calibration mode and when in the Program Mode.
RESET	Functions to unlock height and BMI display.
SEND	Used to “lock” the height and BMI display and to send weight, height and BMI data.

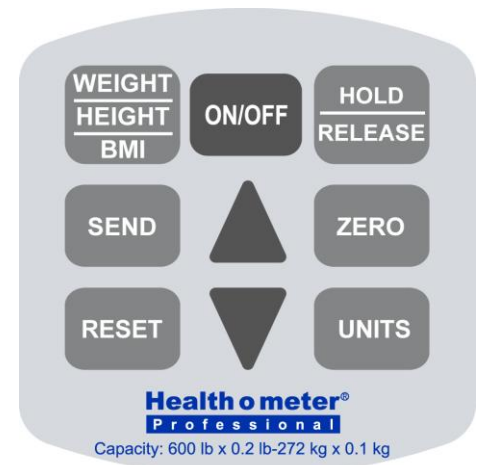CAUTION: To prevent patient injury, the patient must be attended to throughout the entire event.

Weighing a Patient

1. Press the **ON/OFF** button to power the scale on.
2. Wait until '0.0' and 'Zero' appear on the left side of the display.
3. Ask the patient to step on the scale. Ensure they are positioned directly in the center of the platform. The scale will display the patient's weight.

NOTE: The weight will change until the patient weight is stable. Once the patient is standing still, the scale will beep three times and the word “Lock” will appear on the display. The weight may be held on the display by pressing the **HOLD/RELEASE** button. To release, press the **HOLD/RELEASE** button again.

4. If desired, press the **SEND** button to send the weight information to the scale USB port.

NOTE: The scale will always return to the settings and units last used.

OPERATING INSTRUCTIONS (CONT.)

Height Measurement Instructions

WARNING: Use caution when sliding height rod up and down. The rod should move smoothly, do not move the rod in a jerking motion. Fold headpiece down when not in use.

Note: A patient needs to be a minimum of 23-5/8" (60cm) in height for the height to be accurately measured.

1. Press the **WEIGHT/HEIGHT/BMI** button once. The scale will display "----" until the height rod or headpiece is moved. With the headpiece in the folded down position, slide the headpiece up or down to the estimated height of the patient. Then raise the headpiece into its horizontal position.

Note: For patients under 44", slide the headpiece down the pillar.


Patients above 44"

Patients below 44"

Note: When moving the headpiece, ensure the height rod and headpiece are conjoined.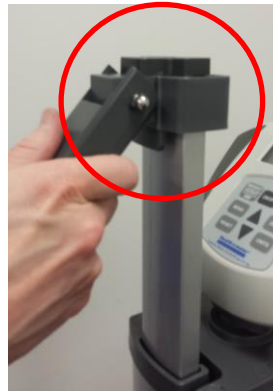

2. Ask the patient to step on the scale platform. Position the patient facing the display head. Ensure that the patient's posture is upright, and that the patient's head is straight and level. Push the headpiece down until it gently rests on the patient's head. The display will show the patient height.

Press the **RESET** button to revert back to weight display.

OPERATING INSTRUCTIONS (CONT.)

Using the Scale for Height Measurement Only

1. Press the **ON/OFF** button to turn the scale on.
2. Wait until '0.0' and 'Zero' appear on the left side of the display.
3. Press the **WEIGHT/HEIGHT/BMI** button once. The scale will display the position of the height rod.
4. Ask the patient to step on the scale platform. Adjust the height rod so that the headpiece gently rests on the patient's head. Read the height.
5. Press the **RESET** button to revert back to weight display.

To Obtain BMI

1. Press the **ON/OFF** button to power on the scale.
2. Ask the patient to step on the scale platform and obtain the patient's weight. Press the **WEIGHT/HEIGHT/BMI** button once. The scale will now display the position of the height rod. While the patient remains on the platform, adjust the height rod so that the headpiece gently rests on the patient's head. Press the **WEIGHT/HEIGHT/BMI** button again. The display will show the patient's BMI.
3. If desired, press the **SEND** button to send the weight, height and BMI information to the scale USB port.
4. Press the **RESET** button to revert back to weight display.

Locking the Weight Measuring Unit in KG (600KL only)

Caution: Please make sure you carefully and precisely follow the directions below as you will be in the PROGRAM MODE of your scale. Any modification outside of what is listed below may greatly affect the functionality of your scale.

1. Press the **ON/OFF** button to power on the scale. As soon as you turn on the scale, press the **ZERO** button once, the **UNITS** button twice, and then the **ZERO** button once again. The scale will display "CAL".
2. Press the ▼ once and "PROG" will show on the display.
3. Press the **SEND** button and OP1-0 or OP1-1 will be displayed.
4. Press the ▼ once and the display will read "OP2-0" for the KG lock function, then press the **UNITS** button to enable KG lock. Your display should read "OP2-1".
5. Press the ▼ until display reads "DONE"
6. Press the **SEND** button, the scale will read "START", then read "0.0 Kg"
You have successfully locked your scale into KG Only.

Deactivating KG Lock (600KL only)

1. Press the **ON/OFF** button to power on the scale. As soon as you turn on the scale, press the **ZERO** button once, the **UNITS** button twice, and then the **ZERO** button once again. The scale will display "CAL".
2. Press the ▼ once and "PROG" will show on the display.
3. Press the **SEND** button and OP1-0 or OP1-1 will be displayed.
4. Press the ▼ once and the display will read "OP2-1" for the KG lock function, then press the **UNITS** button to disable KG lock. Your display should read "OP2-0".
5. Press the ▼ until display reads "DONE"
6. Press the **SEND** button, the scale will read "START", then read "0.0"
You have successfully deactivated the KG lock on your scale.

MAINTENANCE AND CLEANING

General

This section provides instructions for maintenance, cleaning, calibration, and troubleshooting. Maintenance operations other than those described in this section should be performed by qualified service personnel.

1. This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
2. Do not store near chemicals, heaters or radiators.
3. Do not store the scale on its side.

Before first use and after periods of non-use, check the scale for proper operation. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, abuse, etc.
2. Inspect condition of AC adapter cord and prongs, replace if needed.

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:

- mild soap and water solution
- 70% isopropyl alcohol
- solution with 1-5% hydrogen peroxide concentration

After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.

2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

Replacement Parts

Parts may be available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

Part #	Description
600KLKEYPAD	REPLACEMENT KEYPAD
ADPT50	POWER ADAPTER

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated following the procedure below.

Note: Only weights certified and traceable to national standards are to be used for calibration procedures.

KL Model Calibration

KL model scales can be calibrated in pounds or kilograms. The default calibration weight unit is pounds. If you prefer to calibrate in kilograms you must first follow this procedure to change the calibration weight unit.

Caution: Please make sure you carefully and precisely follow the directions below as you will be in the PROGRAM MODE of your scale. Any modification outside of what is listed below may greatly affect the functionality of your scale.

To Change the Calibration Weight Mode to KG

1. Press the **ON/OFF** button to power on the scale. As soon as you turn on the scale, press the **ZERO** button once, the **KG/LB** button twice, and then the **ZERO** button once again. The scale will display "CAL".
2. Press the **▼** once and "PROG" will show on the display.
3. Press the **SEND** button and OP1-0 or OP1-1 will be displayed.
4. Press the **UNITSV** button to toggle between OP1-0 and OP1-1. Select OP1-0 to set your calibration weight in pounds. Select OP1-1 to set your calibration weight to kilograms.
5. After making your selection, press the **▼** until display reads "DONE"
6. Press the **SEND** button, the scale will read "START", followed by a version code and then return to zero.
7. Turn the scale off. Then begin the calibration procedure below.

KG Model Calibration

KG models can only be calibrated in kilograms. Follow the calibration procedure below.

Calibration Procedure for KL and KG Models

The calibration weight must be no less than 60 lb (28 kg) and no more than 300 lb /150 kg. For optimum accuracy, Health o meter Professional Scales recommends calibrating at 300 lb / 150 kg.

OPERATOR ACTION	DISPLAY
Turn on the scale and while "START" is displayed, press the following keys in this sequence: ZERO, UNITS, UNITS, ZERO Note: You must press these keys while "START" is displayed, before the display shows zero.	"CAL"
Press SEND . "Load" and "300.0" or "150.0" will appear on the display.	"Load/300.0" or "Load/150.0"
Press SEND to calibrate with 300 lb or 150 kg. To calibrate with a different amount, press BMI . Use the arrows to change the flashing digit, BMI to move to the next digit, until you reach the required weight (no less than 60lb and no more than 300lb). When the required calibration weight is reached, press SEND . "Clear" will appear on the display.	"CLEAR"
Clear the platform of any weight and press SEND . Dashes will appear while the scale calibrates zero.	"-----"
Wait without touching or shaking the scale until the zero calibration process is finished. The display will then prompt for the calibration weight to be loaded.	"Put XXX.X"
Load the platform with the required weight and press SEND . Dashes will appear on the display as the scale calibrates.	"-----"
When calibration is complete "DONE/CLEAR/XXX" will appear on the display.	"DONE/CLEAR/XXX"
Remove all weight from the platform. When all weight is cleared the scale will reboot and "StArt" will appear on the display followed by the software version and then return to "0.0".	"StArt"; numbers; "0.0"

Refer to the next page should you experience any errors during the calibration procedure.

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting Customer Service at 1-800-815-6615.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Dead battery 2. Faulty electrical outlet 3. Bad power supply (Check LED light at back of display module to determine if adapter cord is working) 	<ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show '0.0' before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value
The display shows "OVER"	The load on the scale exceeds the capacity (600 lb/272 kg)	Remove the excess weight and use the scale according to its limits
The display shows "LO Bat" message	The batteries are depleted	Replace batteries according to instructions

Calibration Errors

ERROR NO.	DEFINITION	CHECK/REPLACE
Err 1	Mathematical mistake; division by zero. Cannot calculate calibration factor	Will show when trying to calibrate with no calibration weight
Err 2	Weight loaded too low. Cannot calibrate	While in calibration procedure add more weight in load mode
Err 3	Unstable weight. Cannot calibrate	Check the load cells mechanical surroundings and see nothing touches them and the cables are well connected
Err 5	Calibration disabled	On the back of the display, locate the silver extended screw and plastic tab. Use a nut driver or pliers to remove the tab and gain access to the push button underneath. Press and release one time to enable proper calibration. When finished, toggle the calibration switch to enable
Err 6	High saturation state (high A/D)	The load cell cable is not connected correctly. Check the connection, cables, and mechanical connection.
Err 7	Low saturation state (low A/D)	See Err 6

HEIGHT ROD TROUBLESHOOTING

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Questionable height	<ol style="list-style-type: none"> 1. Improper connection 2. The display shows “ ____ ” 3. Height is out of tolerance 	<ol style="list-style-type: none"> 1. Check scale base connection, pillar connection, and display head connection – ensure cables are correctly connected 2. Check scale base connection and ensure cables are correctly connected to the display head 3. Check the 3 known height standards. <p>(Dimensions are in cm.)</p> <p>Lowest position – 60.5 +/- 0.8 Neutral position – 112.5 +/- 0.8 Highest position – 222.4 +/- 0.8</p> <p>After checking the 3 known height standards and the connection - determine if the scale needs to be replaced.</p>
The height rod is difficult to move or immobile	<ol style="list-style-type: none"> 1. The pillar may still contain the cardboard within the pillar. 2. Pillar may be damaged. 	<ol style="list-style-type: none"> 1. Look inside the groove of the pillar to ensure the brown cardboard from shipping has been removed. 2. Contact Customer Service at 1-800-815-6615.

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to: Pelstar, LLC

Attention R/A# _____
Return Department
9500 West 55th Street
McCook, IL 60525

Extended Warranty Available

This product is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new products or for a facility's existing product before its current warranty has expired. To learn more, visit www.homscales.com/scalesurance/ or contact your medical supply distributor.

*Not available in all countries.

PELSTAR, LLC

9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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