

PAPER DISPENSER

MODEL 030



1

Use the factory installed inserts to secure the L-brackets into position on the underside of the top. For tables without pre-installed inserts, measure 22" of space between the brackets and mark the small holes.

2

For tables with screw inserts, secure the L-brackets using the (4) four machine screws through the four large holes in the L-brackets and into the inserts. In models without inserts, attach the "L" brackets using the (4) four small wood screws through the four small holes in the L-brackets and into the wood backing of the top.

3

Insert the bar into the slots with the caps on the outside of the mounted brackets.