


eBABYHR DIGITAL BABY HEIGHT ROD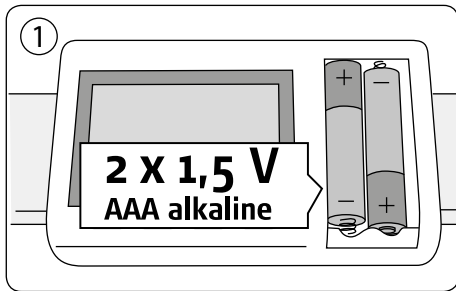


Slide open the battery compartment next to the digital display. Insert 2 AAA batteries and replace compartment cover.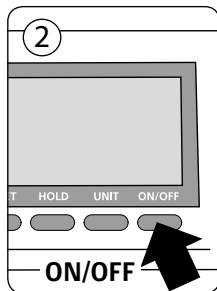

To turn on the digital display, press **ON/OFF**.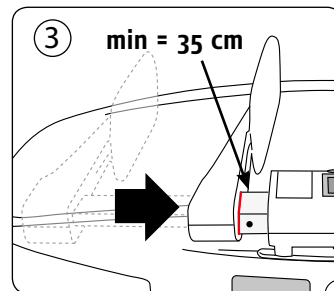


Slide the measuring panel all the way in towards the digital display.

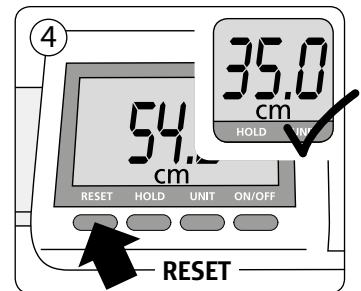Press **RESET** to set the height reference point at 35 cm. This step must be done each time the height rod is powered on.


Place the baby on a flat surface. Position the non-moving panel at the baby's head. Stretch the baby's legs and slide the measuring panel towards the baby's feet.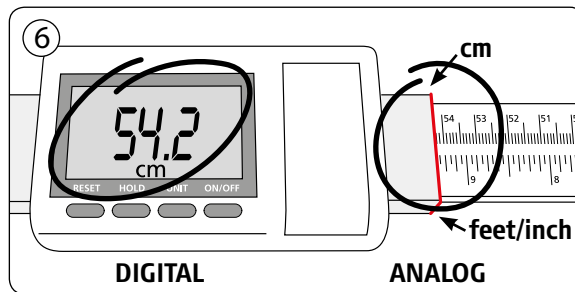


Gently align the measuring panel at the soles of the baby's feet and read the height on the display.

min = 35 cm /
13.78
inches

max = 80,1 cm /
31.54
inches
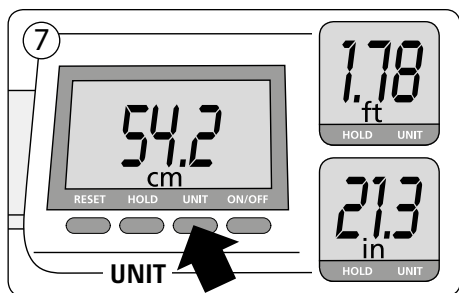



To view the height in another measuring unit, press **UNIT**.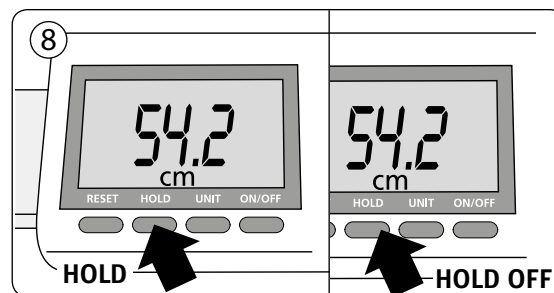

To hold the height on the screen, press **HOLD**. To release the height, press **HOLD** again.

- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- Clean all external surfaces with a clean damp cloth.
- Do not immerse the product into cleaning solutions or water.
- Do not use abrasive cleaners.
- Do not spray the product with liquid.
- In no event whatsoever shall Pelstar, LLC / Health o meter® Professional Scales be liable for damages or injuries arising out of or connected with the assembly, use, or misuse of its products.

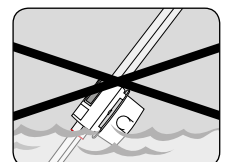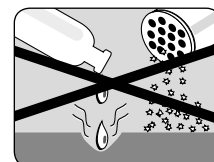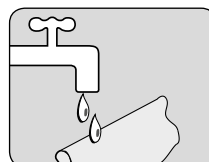eBABYHR DIGITAL BABY HEIGHT ROD

WARRANTY

What does the Warranty Cover?

This Health o meter® Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the product at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction. To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase.

Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 W. 55th St.

McCook, IL 60525

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

© Pelstar, LLC 2018

Health o meter® Professional Scales

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Health o meter® es una marca registrada de Sunbeam Products, Inc. usada bajo licencia.
Los productos Health o meter® Professional son fabricados, diseñados, y de propiedad de Pelstar, LLC.