Model 445KL Mechanical Chair Scale

User Instructions

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

www.homscales.com

© Pelstar, LLC 2016

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Caution and Warning	3
Specifications for This Scale	4
Certifications	5
Unpacking Instructions.....	5
Operating Instructions.....	6
Maintenance	8
Troubleshooting	9
Warranty	10

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTION AND WARNING

- To prevent injury and damage to your scale, please follow these instructions very carefully.
- Assemble and operate the scale per the enclosed user instructions.
- Do not exceed recommended weight limit of 440 lb / 200 kg for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced. Contact your shipper or a Health o meter[®] Professional representative immediately.
- This scale is designed for static weighing of patients only. No scale should be used for patient transfer. Pelstar, LLC assumes no responsibility for patient injury or scale damage should this caution be ignored by the scale operator.
- Do not leave patients unattended while patient is being weighed on the scale.
- Do not stand on footrest. Footrest is not designed to support patient's entire weight.

SPECIFICATIONS FOR THIS SCALE

GENERAL

The Health o meter® Professional 445KL Mechanical Chair Scale is designed to provide accurate, reliable and repeatable weight measurements by providing features that make the weighing process simple, fast and convenient. Weight is displayed in kilograms or pounds. The scale includes a seat with arm supports, and a footstool that can be folded up for easy entry/exit.

445KL SCALE SPECIFICATIONS

Capacity and Resolution	440 lb x .25 lb (200 kg x 0.1 kg)
Environmental	Operating temperatures: 10°C to 35°C (50°F to 95°F) Storage temperatures: 0°C to 50°C (32°F to 122°) Atmospheric Pressure: 50hPa-1060hPa Humidity: 10%-85% RH
Physical Dimensions	Width: 18 1/2" Depth: 17 1/4" Height: 15 3/4"

CERTIFICATIONS / DISPOSAL / SERVICE

Certification Descriptions

**ISO13485:2003
CERTIFIED**

Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

CUSTOMER SERVICE INFORMATION

The 445KL Mechanical Chair Scale is shipped assembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

For further information or telephone support, please contact Customer Service at:

Telephone: 1 (800) 815-6615

Email: HomProCS@homscales.com

445KL UNPACKING INSTRUCTIONS

To avoid damaging the 445KL when unpacking it, **do not use a box cutter, knife, scissors or any sharp object when opening the protective inner packaging.** If damage is found, contact your shipper or a Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the carton as you unpack the parts for assembly. Set the carton aside for storage.

The Health o meter Professional Mechanical Chair Scale is shipped completely assembled, so no additional assembly is required.

Some protective cardboard and plastic will need to be removed before using.

1. Remove metal bands attached to small skid.
2. Remove cardboard at back and front end of chair.
3. Remove protective foam and plastic sleeve over beam.


OPERATING INSTRUCTIONS

Zero Balance Adjustment

The scale should be checked periodically, particularly after being moved, by following the instructions below.

Note: a flat head screwdriver is required.

1. Ensure that the scale is located on a hard, flat, and level surface.
2. Place both poise weights on the “0” value on both bars, as shown below. The beam pointer should float gently up and down, and not touch the top or bottom of the trig-loop opening.
3. If the beam pointer is not centered in the trig-loop opening, use a flat head screw driver to adjust the zero adjustment screw.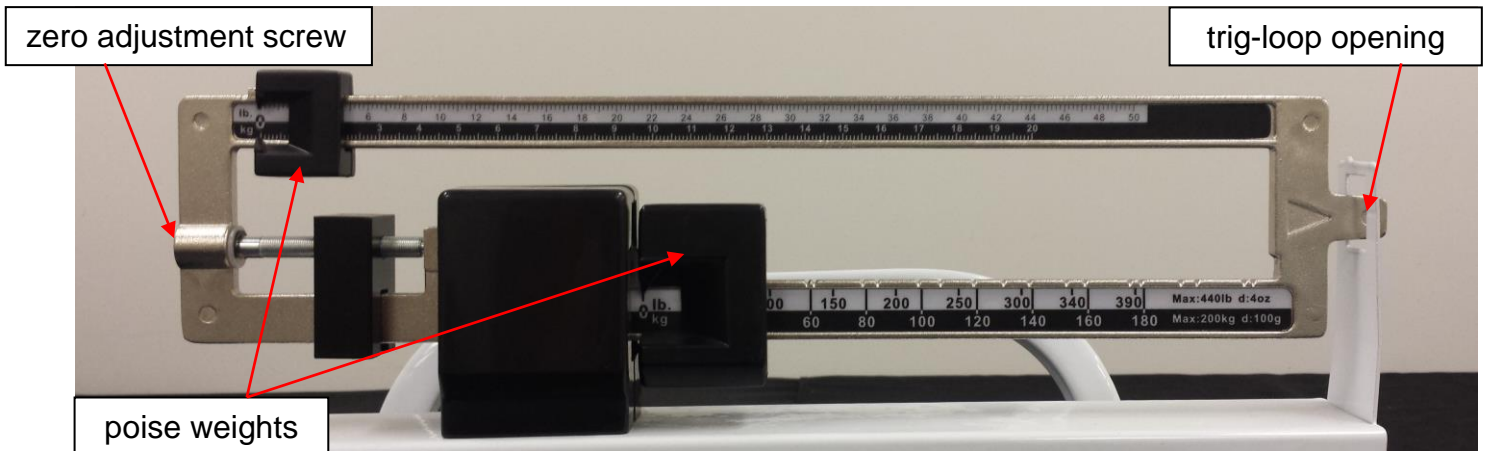

Weighing Instructions

1. Set the large and small poises to indicate the patient's approximate weight.
2. Position the patient onto the scale. Have the patient place their feet on the footrest and not the floor. **Note:** Fold the footrest up before placing the patient onto the scale then fold down when patient is safely seated. When assisting the patient onto the scale, the patient should not use the footrest as a step. Side arms can also be raised to comfortably seat the patient.


OPERATING INSTRUCTIONS CONT.

- Adjust the small and large poise weights until the pointer is centered. **Note:** It is not necessary for you to wait for the pointer to stop moving in order to read the correct weight. As long as the pointer is moving equally within the trig-loop opening, the patient's weight can be determined.

- Observe the location of the indicator arrow built into each poise weight. Read the value on the poise bar, closest to the indicator arrow.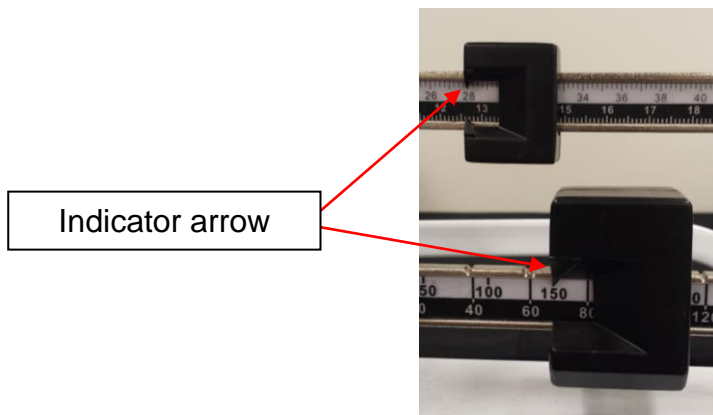

- Add the indicated reading of the small poise value to the indicated reading of the large poise value. The total amount is the patient's weight.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, and troubleshooting for the 445KL Mechanical Chair Scale. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the scale for any obvious damage, wear and tear.

CLEANING


Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

1. Clean all external surfaces with a clean damp cloth. Mild soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the chair scale or arm support in cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol to clean the display surface.
4. Do not use abrasive cleaners.

TROUBLESHOOTING


Refer to the following instructions to check and correct any failure before contacting Customer Service at 1-800-815-6615.

1. In the event that the scale is balanced but still giving inaccurate readings, please gently place the scale on its side to view the assembly under the seat. Make sure the steel rod is connected properly at the base.
2. Make sure that none of the brackets or S shaped loops are turned incorrectly, bent, or damaged underneath the platform.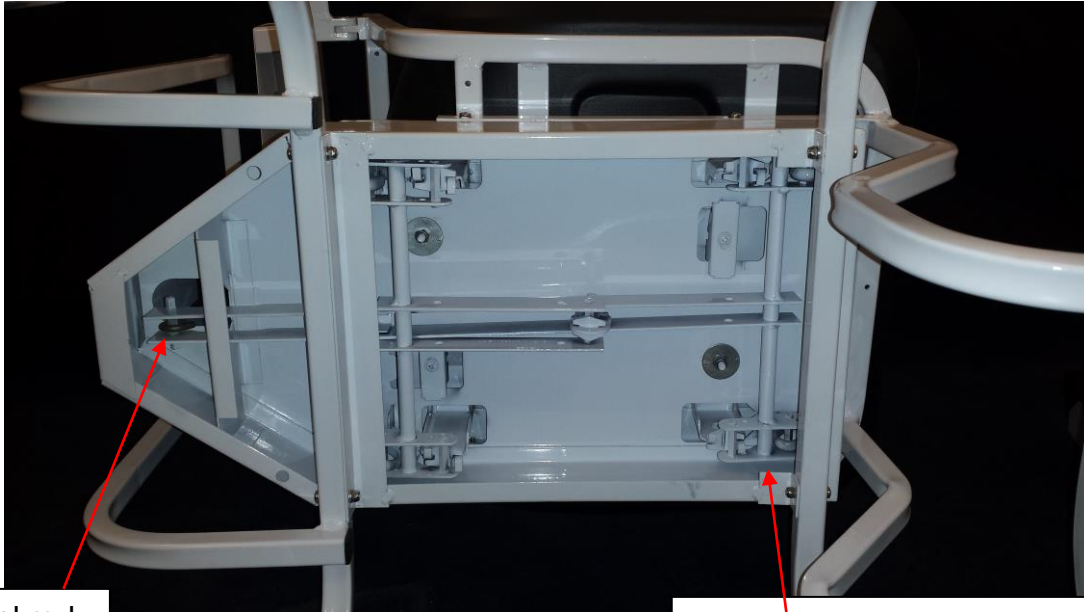

steel rod

brackets and S shaped loops

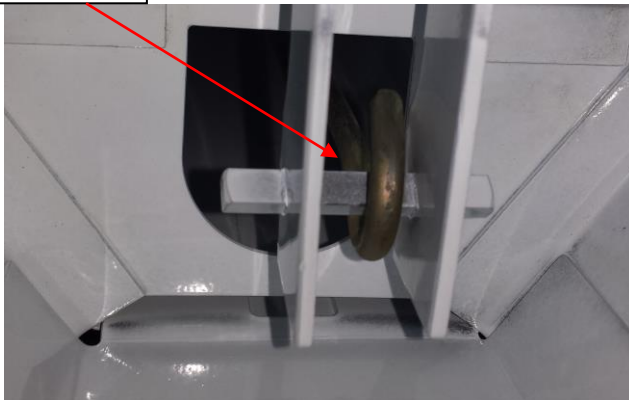WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 W. 55th St.

McCook, IL 60525

PELSTAR, LLC

9500 W. 55th St. • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT: www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Health o meter® est une marque de commerce enregistré de Sunbeam Products, Inc. Utilisée sous licence.
Les produits Health o meter® Professional sont manufacturés, conçus, et possédés par Pelstar, LLC.