

PIG Grippy Surgical Absorbent Mat

Never trip over a hospital blanket again with our revolutionary PIG Surgical Mat with Grippy™ backing.

Features / Benefits:

- **Special Grippy™ backing** — position mat on dry floor before procedures and it stays put no matter what™
- **Highly absorbent** — made with 50% recycled content, our eight-layer pads absorb and retain up to 3 liters to help keep your O.R. floor clean and safe
- **Lies flat on the floor** — no more tripping over bunched-up blankets; also allows for easy movement of mayo stands and other carts
- **Disposable** — save money with fast cleanup and O.R. turnover while avoiding the need to launder heavy, liquid-soaked blankets
- **Superior durability** — mats won't rip, tear or fray under heavy foot traffic
- **Perforated rolls** — easily create smaller pads by tearing at perforations
- **Latex-free**
- **Mobilize your mat** — Add a wheeled, stainless steel mat roll dispenser for easy access to your absorbents

NOTE: PIG Grippy Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

PIG® Grippy® Surgical Absorbent Mat

Item #	Description
HC102	20 Mats • 31" x 40"
HC101	1 Roll • 31" x 100'
HC300	Roll Dispenser



HC101

Available in either rolls or mats

HC102



Flip over for instructions >



New Pig

HOW TO USE


PIG Grippy Surgical Absorbent Mat

PIG Grippy Surgical Absorbent Mat should only be applied to your floor before each procedure and left in place for the duration of the surgery. (If additional absorbents are required during a procedure, use our standard PIG Surgical Absorbent Mat.)

1. Make sure the surface you want to cover is clean, dry and free of debris.
2. Place mat on floor with Grippy™ backing side down.
3. Press down along the edges and into the center of the mat until it is smooth, flat and securely in place, with no air pockets or creases.
4. When the procedure is complete, just pick up the mat and throw it away. (Tip: When picking it up, pinch an edge under the Grippy™ bottom for easy removal.)

WARNING: Used absorbents can take on the hazardous characteristics of liquids absorbed. Follow local, state and federal regulations when storing or disposing of used absorbents.

NOTE: PIG Grippy Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

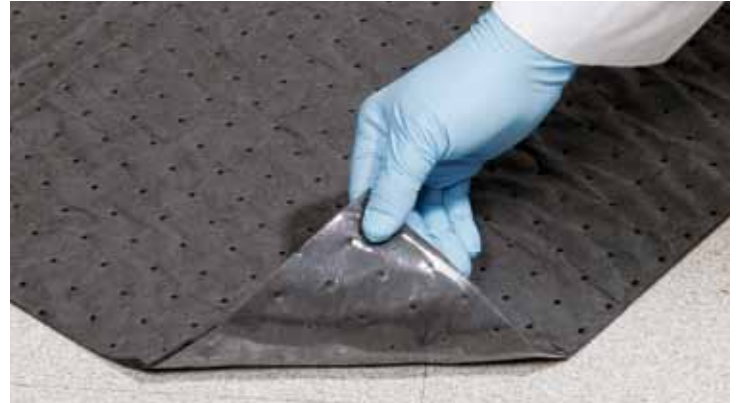

Place with Grippy™ backing side down to eliminate bunching, curling or sliding during procedures.



Grippy® Mat stays put to eliminate tripping and tangled cart wheels.

[Flip over for product details >](#)



New Pig