

## Instructions For Use: Patient Restraint Strap






Product Number: O-PRSLH, O-RSX2

### *Product Features:*

Attaches to siderail quickly.  
Adjusts easily to the patient with cam style buckles.  
Rated for patients weighing up to 500 lbs (227 kg).



### Safety Precautions:

-  **WARNING:** Prior to using this device, please read the instructions for equipment set up and use. Familiarize yourself with the product before application on a patient.
-  **WARNING:** To prevent patient and/or user injury and/or equipment damage, examine the device and surgical-table side rails for potential damage or wear prior to use. Do not use the device: if damage is visible, if parts are missing, or if it does not function as expected.
-  **WARNING:** To prevent patient and/or user injury and/or equipment damage, verify the device hooks (if included) and buckles are firmly secured.
-  **CAUTION:** Do not exceed 500 lbs (227 kg) patient weight.
-  **CAUTION:** To prevent patient and/or user injury and/or equipment damage an authorized specialist must perform all modifications, upgrades, or repairs. Failure to comply may void warranty.

## Instructions for Use:

1. Place the padded area of the Restraint Strap at the desired location on the patient. Drape the Restraint Strap on both sides of the patient.



2. Loop the end of the Strap down between the Surgical Table and the Accessory Rail, then back up on the outside of the Accessory Rail.



3. Lift up on Cam Buckle and thread the Strap through. Pull Strap to adjust tension.



4. To remove, lift up on the Cam Buckle and pull the Strap out from the Buckle. Remove the Strap from the Accessory Rail.

### Cleaning Instructions:

1. After each use, clean the device.
2. Clean and disinfect using a quaternary ammonium disinfecting/cleaning solution following the manufacturer's recommendation for achieving low-level disinfection.
3. Read the cleaning product's directions and follow the instructions on the label. Use caution in areas where fluid migration may occur.
4. Wipe device with a clean, dry cloth.
5. Make certain the product is dry prior to use to avoid damage.



**CAUTION:** Damage may result if the product is cleaned with bleach.