

BD-BB176226
Valve Replacement Kit

1. Carefully remove old valve from stem. Must use a tile Nipper or like instrument to remove metal collar.



DO NOT TWIST red stem as it can tear out.

2. Make sure old valve sheath is removed from red valve stem.

3. Place new metal ring collar on very end of red

valve stem.  **DO NOT PUSH** leave at end.

4. Insert larger half of new valve w/ sheath on into red valve stem. Work in carefully until snug, lubricant will help.

5. Use a snub nose pliers or tile nipper to crimp at least one side of ring collar. Takes some force to crimp.



Do not force anything. If unsure of procedure please return valve and Bean Bag to company for repair of valve.