

Instruction Sheet

Patient Security Cabinet #291600 & #291601

To begin installation of your cabinet open the door using the pre-initialization combination: **C 0π**
Directions for programming your lock are below.

Use the four holes in the rear and appropriate hardware to fasten the cabinet securely to your wall. Place the cap over the top. The cabinet can be mounted ambidextrously by removing the nuts on the back of the lock and reversing the keypad's orientation.

To install the shelf, insert the top tab of each bracket into a rectangular hole in the side wall at your desired height, then snap the lower tab into place. The shelf can be tilted by placing the front brackets as many as three positions below the rear and positioning the shelf with the lip facing up.


Lock Operation

To initialize new locks:

- 1)** Initialize one lock and register the Manager Bypass Key.
Steps 1 and 2 are required only for the first lock, for additional locks go to step 3
 - a. Touch the red Programming key to the key slot for one full second. The LED will turn on.
 - b. Touch each black Manager Bypass Key to the key slot. A single beep and LED blink indicates that the key has been successfully registered.
 - c. Touch the Programming Key to the key slot for one full second. A double tone signifies successful initialization.

- 2)** Copy the Manager Bypass Key registration to the Programming Key
 - a. Press **C 0π 6 6 0π**. The LED will flash
 - b. Touch the Programming Key to the key slot for one full second. A double tone signifies that the Manager Bypass Keys were successfully copied to the Programming Key.

- 3)** Initialize additional locks with the Programming Key
 - a. Touch the programming key to the key slot of each additional lock to be programmed. A double tone indicates that the lock was successfully programmed.

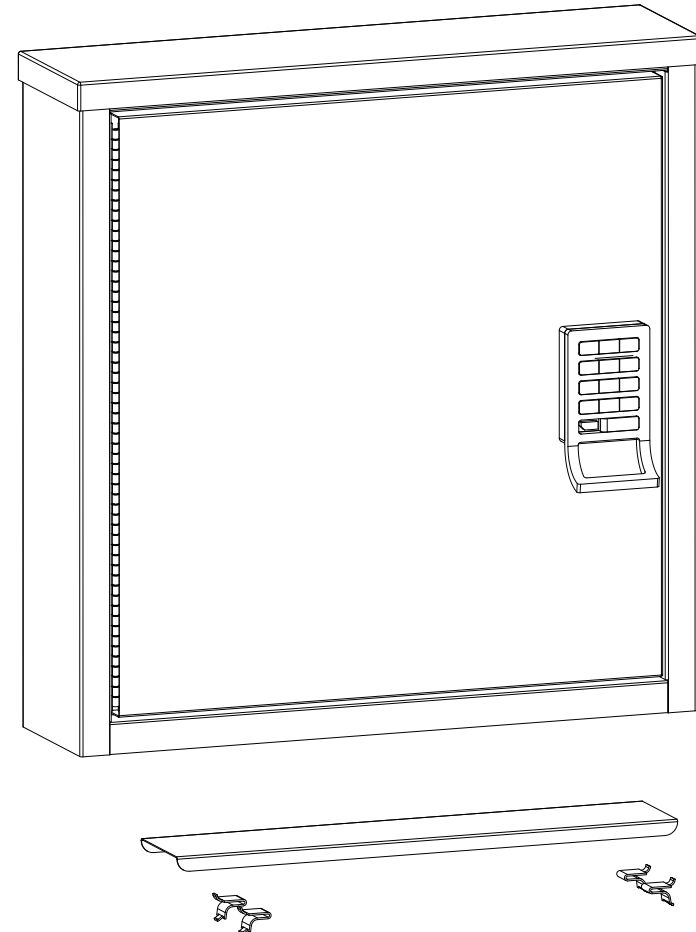

MANUFACTURERS OF **IBeam**® PRODUCTS
OMNIMED INC.®

Instruction Sheet

Patient Security Cabinet #291600 & #291601

To add additional Manager Bypass Keys:

- 1) Press **C** **Omni** **5** **5** **Omni**. The LED will flash.
- 2) Touch the Programming Key to the key slot for one full second. The LED will be on.
- 3) Touch each Manager Bypass Key to the key slot. A single beep and LED blink indicates that the key has been successfully registered.
- 4) Touch the Programming Key to the key slot for one full second. A double tone indicates successful addition of manager keys.
- 5) To add these Manager Bypass Keys to additional locks follow steps 2 and 3 above under "To initialize new locks".

To operate with a user code:

- 1) Press **C** **1** **2** **3** **4** **Omni** To change the user code using the Manager Bypass Key:

To change the user code using the Manager Bypass Key:

- 1) Touch the Manager Bypass Key to the key slot. A beep will sound and the LED will turn on.
- 2) Press **C**.
- 3) Key in the new code followed by **Omni**.
- 4) Press **C**.
- 5) Key in the new code again, followed by **Omni**. The LED will turn off.

To change batteries:

Your lock will emit two sets of three beeps to indicate battery power is low. Note that the Manager Bypass Key will supply interim back-up power.

- 1) Open the door and remove the lock's latch unit cover
- 2) Put four new "AA" cells in the battery holder and replace the cover.

