

Instruction Sheet

Adjustable Box Holders #305320 & #305321

Mounting Adjustable Box Holder

The box holder can be mounted vertically or horizontally using the four mounting holes in the back plate. (mounting hardware not included)

Width Adjustment for Box Holder

Loosen the two screws that secure the box holder frame to the back plate. Slide both halves in or out to conform to the box that you are using. Tighten the screws firmly in place.

Depth Adjustment for Box Holder

Starting with one side of the box holder, use the eraser end of a pencil or similar object to push in the snap button and lift or lower the side support to the desired location of the box. Repeat the step for the opposite site.

Cleaning Adjustable Box Holder

Components can be easily removed for cleaning. Press in snap buttons until side supports release from frame. Loosen the two screws to remove from base plate. Wipe down with soap and water and re-assemble.


Base Plate
P/N: 3099003



Side Support
P/N: 3099002

Frame P/N: 3099007

Snap Button

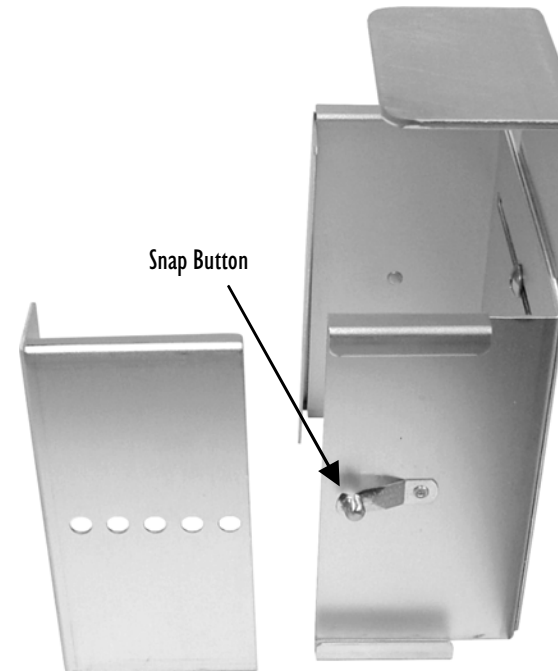

MANUFACTURERS OF **IBEAM**® PRODUCTS
OMNIMED INC.®