Assembly Instructions
Instructions d'assemblage
Montage
Istruzioni di montaggio
Instrucciones de montaje

O-LPA

Product Name Allen® Easy Irrigation Tower
Nom de produit Allen® Tour d'irrigation Facile
Produktname Allen® Irrigationsständer Leicht
Nome prodotto Allen® Deflussore Facile
Nombre del producto Allen® Torre para irrigación fácil

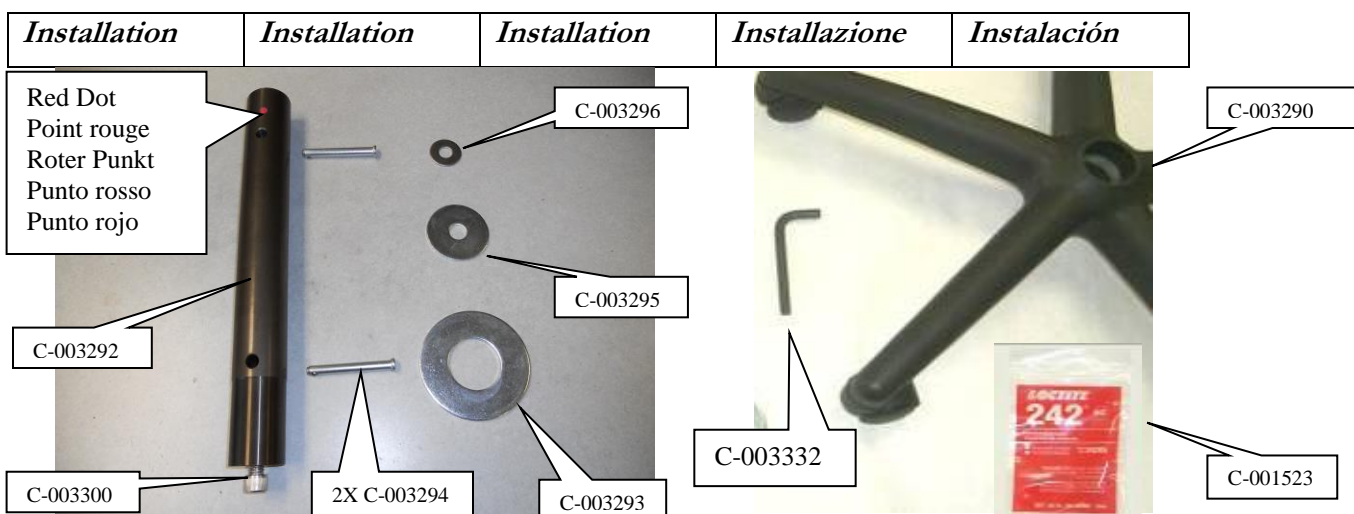
Caution Prior to using this or any other type of medical apparatus, it is recommended that you read the Instructions for Use and familiarize yourself with the product prior to application on a patient.

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Document Number D-720530	Revision A4	Effective Date 10 MAR 2016	©2015 Allen Medical Systems, Inc. All Rights Reserved D-FACT-722-11	Page 1 of 4
-----------------------------	----------------	-------------------------------	---	----------------

Hill Rom S.A.S
 BP. 14 Z.I. du Talhouet
 56330 Pluvigner, France
 Tel. +33 (0) 2 97 50 92 12
 Fax +33 (0) 2 97 50 92 00

Step 1: Insert bar (C-003292), the end with the red dot, into the center square-shaped cylinder of the Tower with the holes facing up. Align holes and red dots in Tower and bar.

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Step 2: Install washer (C-003293) onto the end of the bar.

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
Step 3: On the side with the Red Dot, insert Lock Pin (C-003294) through the upper hole at the bottom of the tower and the corresponding hole in the bar. Install second pin in lower hole. NOTE: Assure that the head of the Loc Pins are fully installed and flush with the surface of the hole in the Tower.

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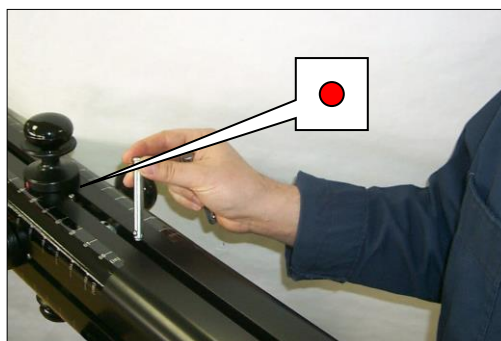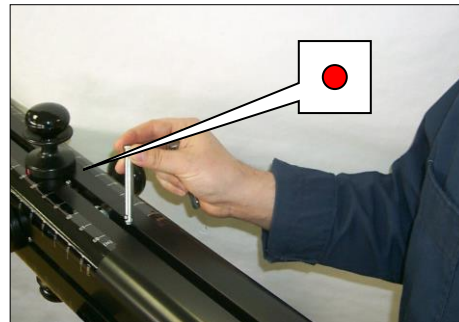
Step 4: Install Base Unit (C-003290) onto the end of the Tower as illustrated. **NOTE:** The Base Unit may not fully engage at this point. Tightening the screw in Step 6 will bring the bottom surface of the bar flush with the surface of the larger washer.

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Step 5: Place smaller washer (C-003296) onto screw (C-003300) followed by the larger washer (C-003295). Apply Loctite 242 (provided) to the threads of screw (C-001523) as illustrated.

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Step 6: Using the Allen wrench (C-003332) (provided), thread washer and screw assembly onto base of Tower until washer is flush tightly against the surface of the bar.

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