

Allen[®] Easy Irrigation Tower O-LPA



WARNING

Prior to using this or any other type of medical apparatus with a patient, it is recommended that you read the *Instructions for Use* and familiarize yourself with the product.





INSTRUCTIONS FOR USE

IMPORTANT NOTICES



READ THESE INSTRUCTIONS COMPLETELY BEFORE USING THIS PRODUCT.

- Read and understand all warnings in this manual and on the device itself prior to use with a patient.
- The  symbol is intended to alert the user to important procedures or safety instructions regarding the use of this device.
- The  symbol on the labels is intended to show when the IFU should be referenced for use.
- The techniques detailed in this manual are only manufacturer's suggestions. The final responsibility for patient care with respect to this device remains with the attending physician.
- Device function should be checked prior to each usage.
- This device should only be operated by trained personnel.
- All modifications, upgrades, or repairs must be performed by an authorized specialist.
- Keep this manual available for future reference.



INSTRUCTIONS FOR USE

SET UP AND USE

1. Hang bag on any of the four hooks.
2. Squeeze handle and move bag up or down to desire height.
3. Release handle and check to ensure that the handle is locked into place.



CLEANING AND MAINTENANCE

- After each use, clean the device.
- Clean and disinfect using a quaternary ammonium disinfecting/cleaning solution following the manufacturers recommendation for achieving low-level disinfection.
- Read the cleaning product's directions and follow the instructions on the label. Use caution in areas where fluid migration may occur.
- Wipe device with a clean, dry cloth.
- Make certain the product is dry prior to reinstalling to avoid damage.



WARNING: Damage may result if product is cleaned with bleach.

For ordering information please see catalog. All modifications, upgrades, or repairs must be performed by an authorized specialist.

NORTH AMERICA

(800) 433-5774

(978) 263-8846 FAX

INTERNATIONAL

001 978 266 4200 ext. 4286

001 978 263 5230 FAX

Document Number
D-729004

Revision
A8

Effective Date
OCT 11, 2013

©2013 Allen Medical Systems, Inc.
All Rights Reserved

D-FACT-722-09

Page
3 of 3



Hill Rom S.A.S
BP. 14 Z.I. du Talhouet
56330 Pluvigner, France
Tel. +33 (0) 2 97 50 92 12
Fax +33 (0) 97 50 92 12

