



Cardinal[®]
DETECTO

HS25KGP
Suspended Baby Scale
Owner's Manual



8535-M214-O1 Rev B
03/15

Printed in USA

UNPACKING

1. Open box.
2. Remove top Styrofoam insert from box.
3. Next, remove S-hook from insert and set aside.
4. Grasp scale and remove from box.
5. Remove Pants (baby sling seat) from box.
6. Remove bottom Styrofoam insert from box.
7. Turn insert over to locate second S-hook.
8. Remove S-hook from insert and set aside.
9. Remove scale from plastic bag.
10. Remove cardboard dowel between scale case and lower loop.
11. Scale is now ready to use.



Figure No. 1

OPERATION

1. Referring to Figure No. 1, install S-hooks on scale.
2. Unfold Pants (Baby Sling Seat).
3. Hang Pants on lower S-hook. See Figure No. 1.
4. Referring to Figure No. 2, turn Zero Adjust Knob until dial pointer is reading "0".
5. Place baby in Pants.
6. The dial pointer will indicate the baby's weight.
7. Remove baby from Pants and re-zero scale if necessary.

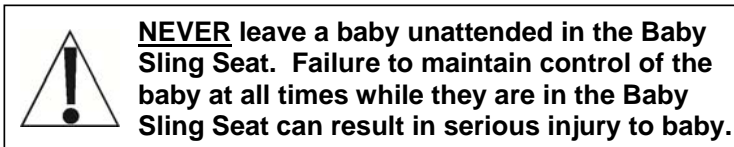


Figure No. 2

CARE AND CLEANING

- **DO NOT** spray; water directly on the scale.
- **DO** clean scale using a damp, soft cloth and mild detergent.
NOTE! Some types of paper towels may scratch the dial lens.
- **DO NOT** use acetone, alcohol, or other volatile solvents for cleaning.

PARTS LIST

ITEM	QTY	PART NUMBER	DESCRIPTION
1	4	6600-1870	Lens Screw
2	1	6600-1871	Lens
3	2	6600-1872	S-Hook
4	1	HSPANT	Pants (Baby Sling Seat)