

Disposable Liner & Receptacle System

INSTRUCTIONS FOR USE



1 Put roll of Liners in Receptacle feeding from the top




2 Pull Liner until it reaches Perforation



3 Press down gently on Receptacle lid with one hand



4 Tear gently across perforation

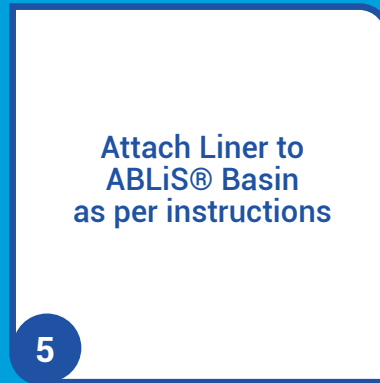

5 Follow IFU for ABLiS

Please keep a copy of IFU - this must be retained for future reference.

CMCP 10001A - PathAguard® Reusable Basin

CMCL 10001A - PathAguard® Single Use Liner

CMCD 10001A - PathAguard® Liner Receptacle

